

GMP
ISO9001
ISO14001
ISO13485
CERTIFIED

PIHSIANG®



MODEL: ***TE-888iX***

(PATENTED)

PIHSIANG MACHINERY MFG. CO., LTD.


***NO.108, HSIN-HE ROAD, HSIN-FENG HSIANG 304
HSINCHU COUNTY, TAIWAN***

MODEL :


[illegible]

TABLE OF CONTENTS ILLUSTRATION

INDEX.1	FRONT CHASSIS ASSEMBLY-----	1-1~2
INDEX.2	REAR CHASSIS ASSEMBLY-----	2-1~2
INDEX.3	DRIVE-TRAIN ASSEMBLY-----	3-1
INDEX.4	FRONT AXLE ASSEMBLY-----	4-1~3
INDEX.5	P-TYPE HANDLE BAR W/ HAND GRIP-----	5-1
INDEX.6	DELUXE SEAT ASSEMBLY-----	6-1
INDEX.7	FRONT CHASSIS COVER ASSEMBLY-----	7-1~2
INDEX.8	REAR CHASSIS COVER ASSEMBLY-----	8-1~2
INDEX.9	HANDLE BAR COVER ASSEMBLY-----	9-1~2
INDEX.10	UPPER CONTROLLER ASSEMBLY-----	10-1~2
INDEX.11	P-TYPE HANDLE BAR ASSEMBLY-----	11-1
INDEX.12	MAIN CONTROL BOX ASS'Y-----	12-1
INDEX.13	W/ BATTERY-----	13-1
INDEX.14	CONTROL AND BATTERY WIRING ILLUSTRATION-----	14-1
	APPENDIX (1) - WIRING DIAGRAM-----	
	APPENDIX (2) - STICKER AND DECAL-----	

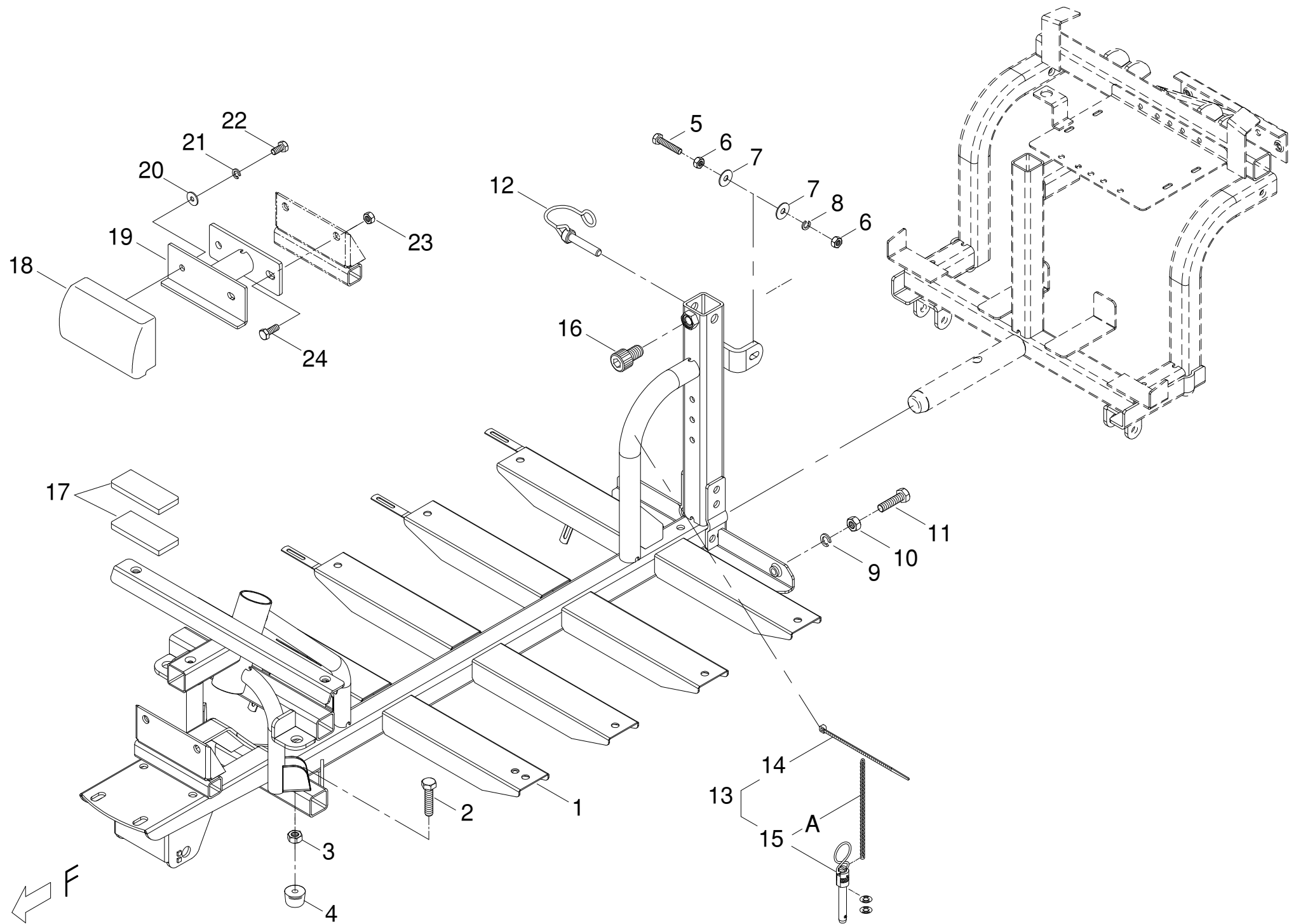


Index NO.	Part NO.	Qty.	Description	Specification	Remark
1	-----		Front chaaais assembly		
1-1	102101-88103	1	Front Chassis	888iX	
			-----*-----		
1-2	5001-08016-210	2	Hex head screw	M8*P1.25*16L,Multicolored Galvanized	
1-3	5011-08013-200	2	Hex Nut	M8*P1.25*13,Multicolored Galvanized	
1-4	209915-89500	2	Rubber	889DX	
			-----*-----		
1-5	5001-06030-210	2	Hex head screw	M6*P1.0*30L,Multicolored Galvanized	
1-6	5011-06010-200	4	Hex Nut	M6*P1.0*10*5L,Multicolored Galvanized	
1-7	5021-06018-210	4	Flat Washer	§ 6*18*1t,Multicolored Galvanized	
1-8	5022-06010-210	2	Spring Washer	§ 6*10, Multicolored Galvanized	
			-----*-----		
1-9	5022-08013-210	2	Spring washer	§ 8*13,Multicolored Galvanized	
1-10	5011-08013-200	2	Hex Nut	M8*P1.25*13,Multicolored Galvanized	
1-11	5001-08030-210	2	Hex head screw	M8*P1.25*30L,Multicolored Galvanized	
			-----*-----		
1-12	202922-88205	1	Connecting pin	787N	
			-----*-----		
1-13	-----		Safety pin assembly		
1-14	505201-01200-50	1	cable Tie	CV-120B	
1-15	202922-00102	1	Connecting pin	§ 7.93,With two loops	
1-A	-----	REF	Chain		
			-----*-----		
1-16	5002-10016-720	1	CAP HEAD ALLEN SCREW	M10*P1.25*16L,Black Galvanized	

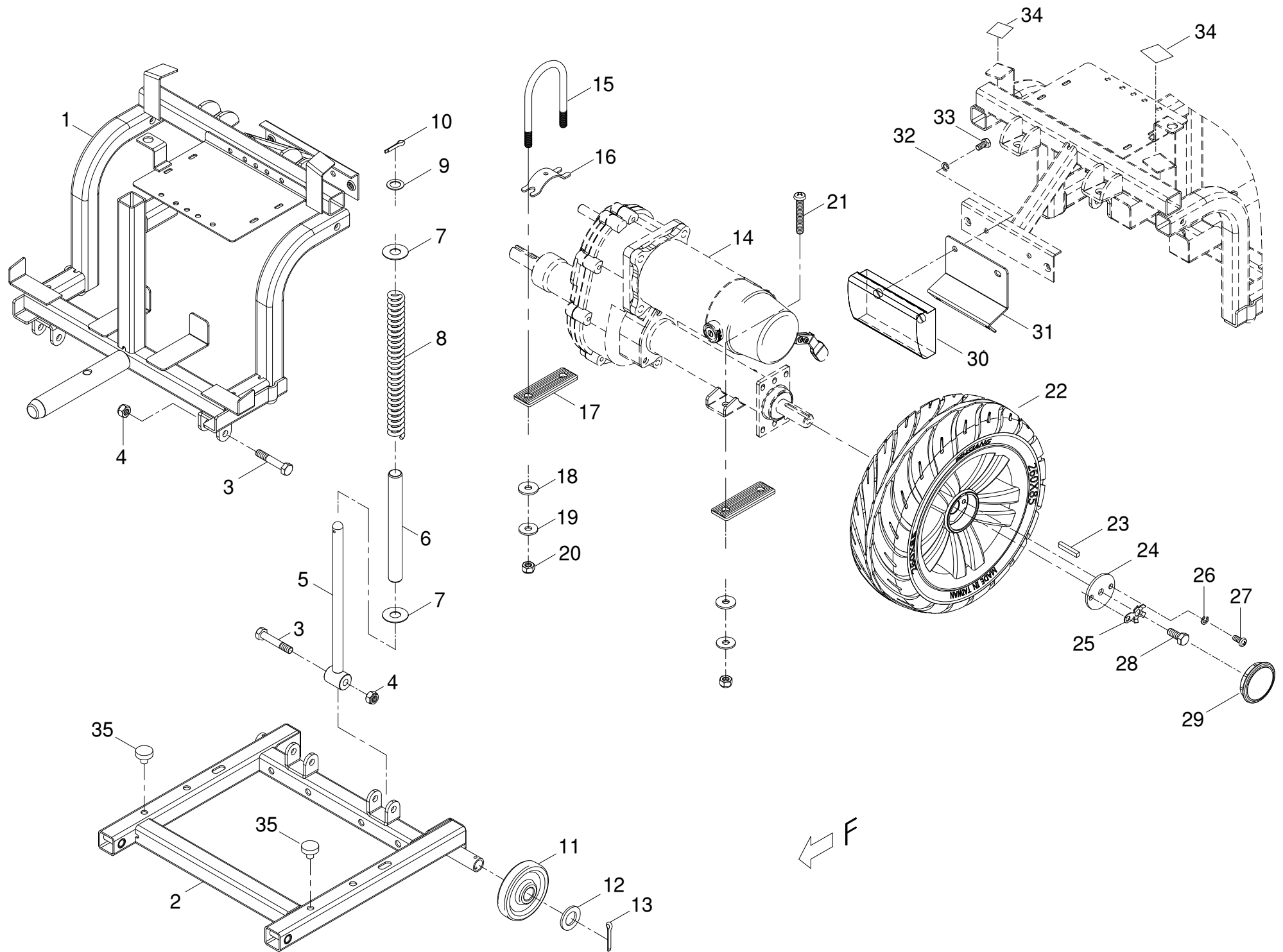


Index NO.	Part NO.	Qty.	Description	Specification	Remark
2	-----		Rear chassis assembly		
2-1	102202-88101-A2	1	Rear Chassis	Upper, 888iX, A2	
2-2	102203-88100-A4	1	Rear Chassis	Lower #1, 888NRS/888iX, A4	
			-----*-----		
2-3	5001-08045-220	4	Hex Head Bolt	M8*P1.25*45L,Multicolored Galvanized	
2-4	5012-08013-200	4	Nyloc Nut	M8*P1.25*13 7.8t,Multicolored Galvanized	
2-5	102907-80101	2	Connecting rod	210L	
2-6	4002081-01	2	tube	B81-1	
2-7	4001406-03	4	Flat Washer	A406-3, ϕ 12.5*30*1t	
2-8	202929-88101	2	Spring	ϕ 4WD* ϕ 18*210L,34.2 Circles	
2-9	4001406-02	2	Flat Washer	A406-2, ϕ 12.5*20*1t	
2-10	5023-03020-440	2	Split pin	M3*20L,SUS	
			-----*-----		
2-11	209928-88100	2	anti-tip wheel	3"	
2-12	4001305-00	2	Flat Washer	A305, ϕ 17.5*30*2.6t	
2-13	5023-03025-210	2	Split pin	M3*25L,Multicolored Galvanized	
			-----*-----		
2-14	-----	REF(1)	Drive-train Assembly		
2-15	4003210-03-A4	1	U-Type Screw, Black Galvanized	C210-3-A4, 787L	
2-16	4001192-01	1	Arc Plate	A192-1,Black	
2-17	209909-69400	2	Rubber Gasket	81*26*4.75t(GK7)	
2-18	209908-69200	4	Rubber Washer	787L	
2-19	5021-08022-710	4	Flat Washer	ϕ 8*22*2t,Black Galvanized	
2-20	5012-08013-700	2	Nyloc Nut	M8*P1.25*13,7.8t,Black Galvanized	

Index NO.	Part NO.	Qty.	Description	Specification	Remark
2-21	5005-08050-710	2	Philip Screw	M8*P1.25*50L,Black Galvanized	
			-----*-----		
2-22	105201-88109	2	Rear Wheel Assembly	10", Black Tire, Silver Rim	
2-23	505301-00505-02	2	Drive Key	5*5*35	
2-24	4001364-00	2	Flat Washer	A364, § 8*45*3t	
2-25	4001481-00	2	Locking tab washer	A481	
2-26	5022-06010-210	4	Spring Washer	§ 6*10, Multicolored Galvanized	
2-27	5005-06012-440	4	Screw	M6*P1.0*12L,SUS	
2-28	5001-08020-440	2	Hex Head Bolt	M8*P1.25*20L,SUS304	
2-29	203512-88102	2	Rim Cover	Silver	
2-30	209933-80500	1	PU cushion		
2-31	202934-88100	1	Bracket	888iX	
2-32	5022-06010-210	2	Spring Washer	§ 6*10, Multicolored Galvanized	
2-33	5001-06016-210	2	Hex head screw	M6*P1.0*16L,Multicolored Galvanized	
2-34	203103-66500	REF	Velcro	25W*25M,Hook	
2-35	209915-00101	2	Rubber Bump Stop	UL7-WR	
-----	301155-92	1	CE Label	N55-92, UK18-888iX	
-----	-----	REF(1)	Decal		

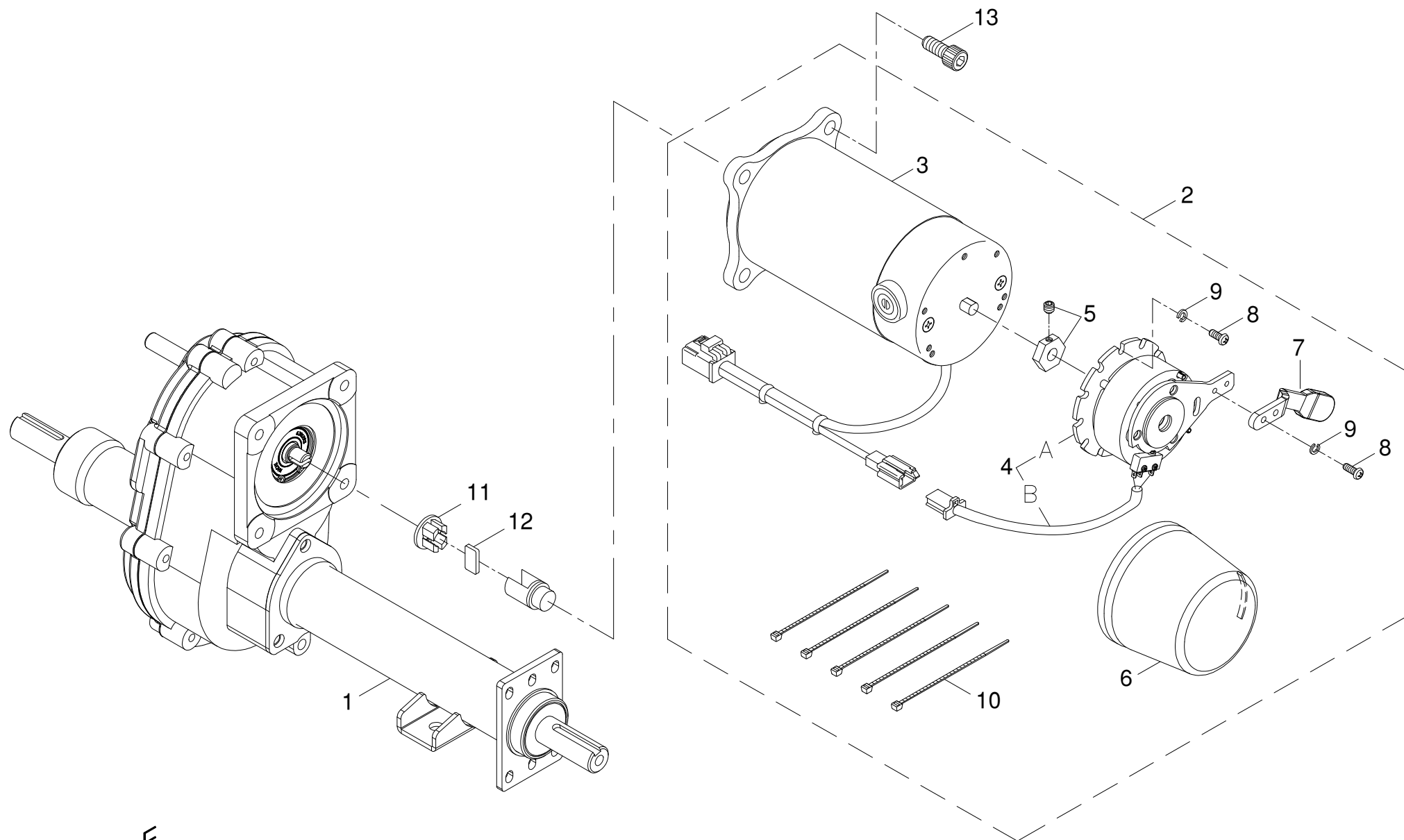


Index NO.	Part NO.	Qty.	Description	Specification	Remark
3	100001-88113	1	Drive-train Assembly	8MNF-3, #3, #1-1, A380-24	
3-1	100401-88103-A6	1	Transaxle	888NTBG(#1-2), Black, A6	
3-2	100101-88118	1	Motor and Brake Assembly	8MNF-3, #3 (A380-24)	
3-3	100202-88106-A2	1	Motor assembly	8MNF-3, A2	
-----	200203-88104-A2	2	Carbon brush		
-----	200203-88105	2	Carbon brush cover		
-----	-----	REF(2)	Decal		
3-4	100303-00108	1	Electromagnetic Brake Assembly	#3E-A2,WR229P1,UL7-W-L(A2)	
3-A	100306-00100-A4	1	Electromagnetic Brake	#3, WR198A408P1, A4	
3-B	2370-00503-02	1	Connection Cable	UL7-W-L3, 210L	
-----	9004-001	1	Heat shrink sleeve	F3,3*0.25	
-----	9004-012	1	Heat shrink sleeve	F32-6,6*0.25*100M	
3-5	200302-88101-A2	1	Driving Hex Axle for Brake Disc	#2-1, 787N, A2	
3-6	2399-00201-00	1	Protect Cover for Magnetic Brake	UL7-W/WD	
3-7	4001380-24	1	Extension Lever for Freewheel	A380-24, for Brake #10 on FS888	
3-8	5005-04010-210	5	Philip Screw	M4*P0.7*10L,Multicolored Galvanized	
3-9	5022-04007-210	5	Spring washer	§ 4*7*1.25t,Multicolored Galvanized	
3-10	505201-00940-50	5	cable Tie	CV-094	
3-11	200904-88100	1	Joint Adapter		
3-12	200905-88100	1	Plate for Joint Adapter		
3-13	5002-08020-220	4	CAP HEAD ALLEN SCREW	M8*P1.25*20L,Multicolored Galvanized	
-----	301179-04	1	Trademark Sticker	N79-4	
-----	-----	REF(1)	Decal		






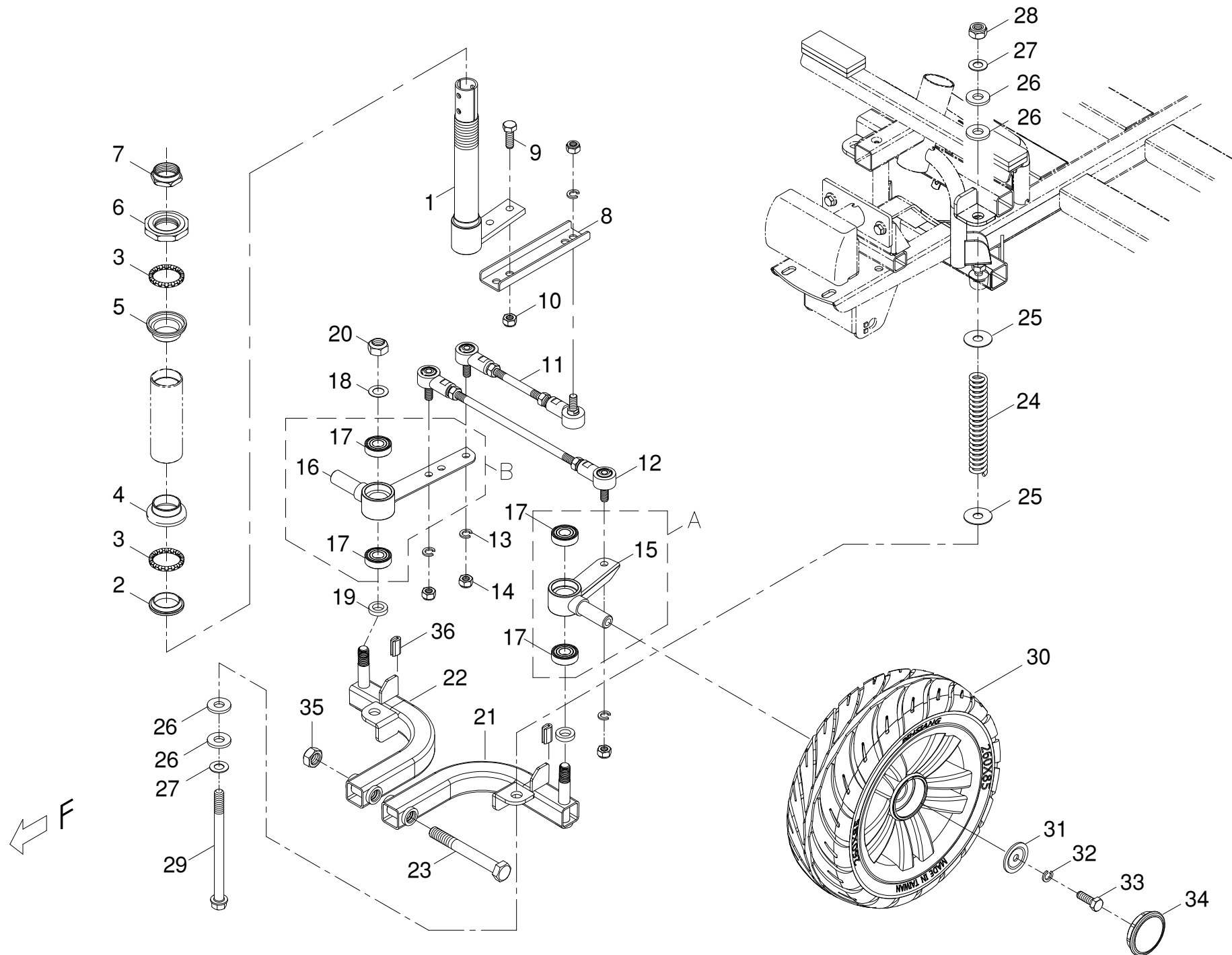


Index NO.	Part NO.	Qty.	Description	Specification	Remark
4	-----		Front axle assembly		
4-1	101501-88100	1	Steering Shaft Assembly	888E/N/NR, for T/P/Delta Tiller	
			-----*-----		
4-2	201401-88200	1	Bearing bushing	§ 27	
4-3	201402-88200	2	Steering Bearings	5/32*16	
4-4	201403-88200	1	Bearing housing	§ 30	
4-5	201404-88200	1	Bearing housing	§ 30	
4-6	201405-88200	1	Bearing bushing	Large octangle	
4-7	201406-88200	1	Steering Head Nut	Dia 22.2	
			-----*-----		
4-8	4001035-00	1	U Type connecting plate	A35	
4-9	5001-08020-210	2	Hex head screw	M8*P1.25*20L,Multicolored Galvanized	
4-10	5012-08013-200	2	Nyloc Nut	M8*P1.25*13 7.8t,Multicolored Galvanized	
			-----*-----		
4-11	101601-88100	1	Connecting rod assembly (Short)	85L	
-----	201601-88101	1	Connecting rod	M8*P1.25*85L,Galvanized	
-----	5011-08013-200	2	Hex Nut	M8*P1.25*13,Multicolored Galvanized	
-----	5022-08013-210	2	Spring washer	§ 8*13,Multicolored Galvanized	
-----	201602-88100	2	Link Ball		
			-----*-----		
4-12	101701-88100	1	Connecting rod assembly (Long)	220L	
-----	201701-88101	1	Connecting rod	L,M8*P1.25*220L,Multicolor	
-----	5011-08013-200	2	Hex Nut	M8*P1.25*13,Multicolored Galvanized	
-----	5022-08013-210	2	Spring washer	§ 8*13,Multicolored Galvanized	

Index NO.	Part NO.	Qty.	Description	Specification	Remark
-----	201602-88100	2	Link Ball		
			-----*-----		
4-13	5022-08013-210	4	Spring washer	§ 8*13,Multicolored Galvanized	
4-14	5012-08013-200	4	Nyloc Nut	M8*P1.25*13 7.8t,Multicolored Galvanized	
			-----*-----		
4-A	102903-88100-A3	1	Front wheel axle assembly	Left,Short	
4-15	202902-88101	1	Front wheel axle	Short-FCD,Black,Left	
4-17	505101-60012-01-1	2	Ball bearing	#6001zz,NBN	
			-----*-----		
4-B	102904-88100-A3	1	Front wheel axle assembly	Right,Long	
4-16	202903-88101-A2	1	Front wheel axle	Long-FCD,Black,Right	
4-17	505101-60012-01-1	2	Ball bearing	#6001zz,NBN	
			-----*-----		
4-18	5021-12021-210	2	Flat Washer	§ 12*21*0.5t,Multicolored Galvanized	
4-19	4005006-00	2	Spacer	H6, § 12ID* § 20OD*4.5T	
4-20	5014-12019-210	2	Self-lock nut	M12*P1.25*19,Multicolored Galvanized	
4-21	101802-88100-A2	1	Front Crossbeam	Left, 888NRS, A2	
4-22	101803-88100-A2	1	Front Crossbeam	Right, 888NRS, A2	
4-23	5001-12100-220	1	Hex Head Bolt, 8.8 Graade	M12*P1.25*100L, Multicolored Galvanized	
4-24	202928-88100	2	Spring	§ 4WD* § 17.5*120L,19 Circles	
4-25	4001406-01	4	Flat Washer	A406-1, § 10.1*30*1t	
4-26	209908-89500	8	Rubber Gasket	Round, § ID11* § OD25*3T	
4-27	4001406-00	4	Flat Washer	A406, § 10.1*20*1t	
4-28	5012-10017-200	2	Nylon lock nut	M10*P1.25*17,Multicolored Galvanized	

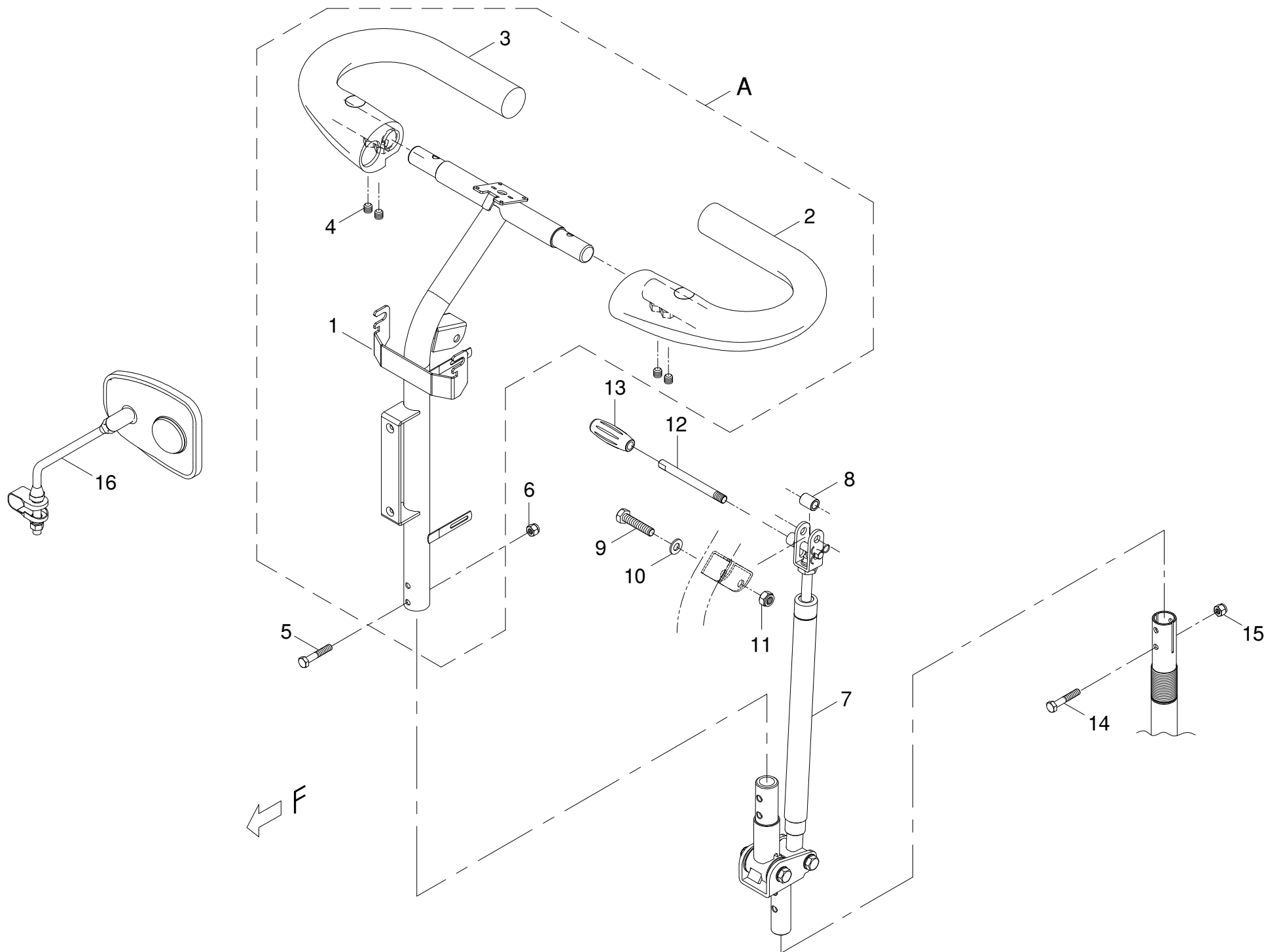

Index NO.	Part NO.	Qty.	Description	Specification	Remark
5-A	101104-88103	1	Handle Bar, w/ PU Coating Hand Grip	888iX, P-type Tiller	
5-1	101101-88103	1	Loop Handle Frame	888iX	
5-2	101906-80501	1	Loop handle bar	Left,888A#1,Delta tiller	
5-3	101907-80501	1	Loop handle bar	Right,888A#1,Delta tiller	
5-4	5004-08008-440	4	Set Screw	M8*1.25*8L, SUS	
			-----*-----		
5-5	5001-06035-720	2	Hex head screw	M6*P1.0*35L,Black Galvanized	
5-6	5012-06010-702	2	Nylon lock nut	M6*P1.0*10*8t,Black Galvanized	
			-----*-----		
5-7	800907-881-13	1		w/ Gas Lift	
5-8	4003264-61	1	Spacer	C264-61	
5-9	5001-08040-710	1	Hex Head Screw	M8*P1.25*40L, Black Galvanized	
5-10	5021-08016-710	1	Flat Washer	8*16*1.5T,Black Galvanized	
5-11	5012-08013-700	1	Nyloc Nut	M8*P1.25*13,7.8t,Black Galvanized	
5-12	202912-89100-A2	1	Operating Stem	100L, for P Bar, A2	
5-13	209929-88801	1	Rubber Cap for Lever of Sliding Seat	ID § 8.1*50.7L, Black	
			-----*-----		
5-14	5001-06035-720	2	Hex head screw	M6*P1.0*35L,Black Galvanized	
5-15	5012-06010-702	2	Nylon lock nut	M6*P1.0*10*8t,Black Galvanized	
5-16	202908-88102-A3	2	Back Mirror for T/P/C-bar	KS-800810, A3	

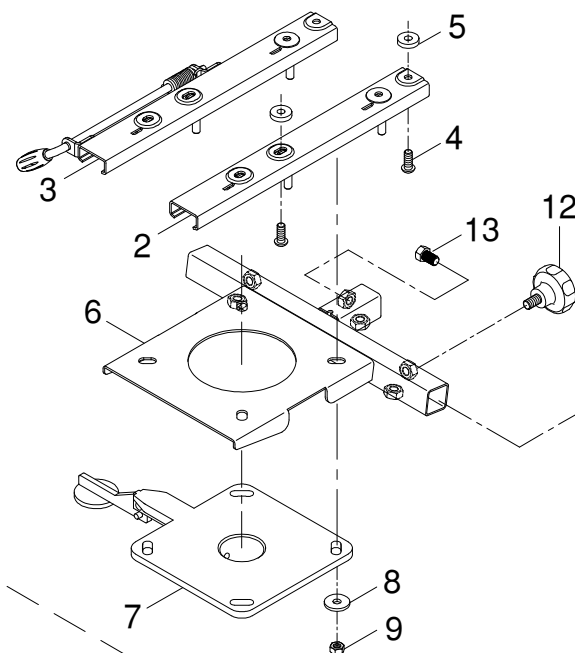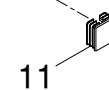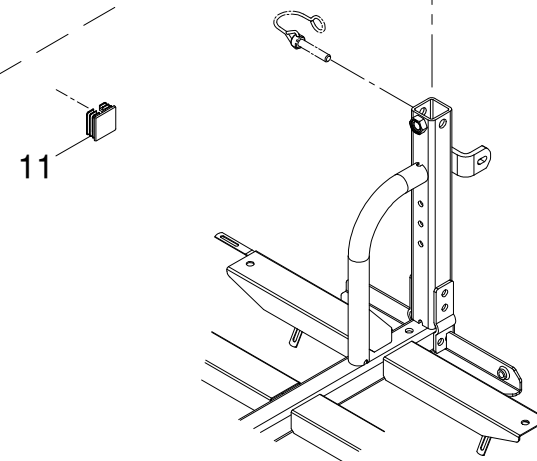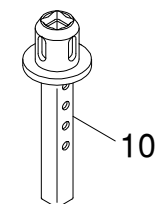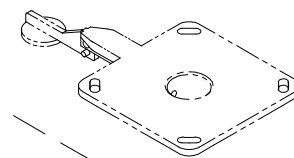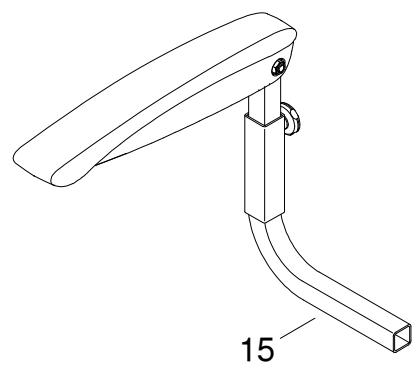


A

INDEX.6


[back to contents](#)

Index NO.	Part NO.	Qty.	Description	Specification	Remark
6-A	800317-778-18	1	Black Vinyl Deluxe Seat Assembly	w/ Slide & Reclining to 112 Degrees	
6-1	104021-00112	1	Black Vinyl Captain Seat	18.5"x17.5",w/Reclining 112,Headrest	
6-2	204901-88803-A2	1	Sliding Rail	Left, #5, A2	
6-3	204902-88803-A2	1	Sliding Rail	Right, #5, A2	
6-4	5003-08020-221	4	Allen Button Head Screw	M8*P1.25*20L,Multicolored Galvanized	
6-5	5021-08022-211	4	Flat Washer	Dia 8*22*5t,Multicolored Galvanized	
6-6	204506-66101	1	Seat Adapter	#2	
6-7	204507-66103	1	Swivel Seat Stand	#4	
6-8	4001428-00	4	Flat Washer	A428, \$ 8.4*26*3t	
6-9	5012-08013-200	4	Nyloc Nut	M8*P1.25*13 7.8t,Multicolored Galvanized	
6-10	204508-88805-A2	1	Seat Post Adaptor	165L, A2, w/ Hole	
6-11	209931-00105	2	Square Plug	30MM*30MM,2T	
6-12	5008-10020-212-A2	2	Tighten Knob	M10*17L,Multicolored Galvanized,A2	
6-13	5001-10016-710	1	Hex Head Screw, Nyloc	M10*P1.25*16L, Black Galvanized	
			-----*-----		
6-14	104300-00106-A5	1	Flipback Height Adjustable Armrest	Left, Econ., 14"H, Wid.58, A5, 92° Bent	
6-15	104400-00106-A5	1	Flipback Height Adjustable Armrest, 92°	Right, Econ., 14"H, Wid.58, A5	
-----	301174-31	1	Caution Sticker	N74-31	

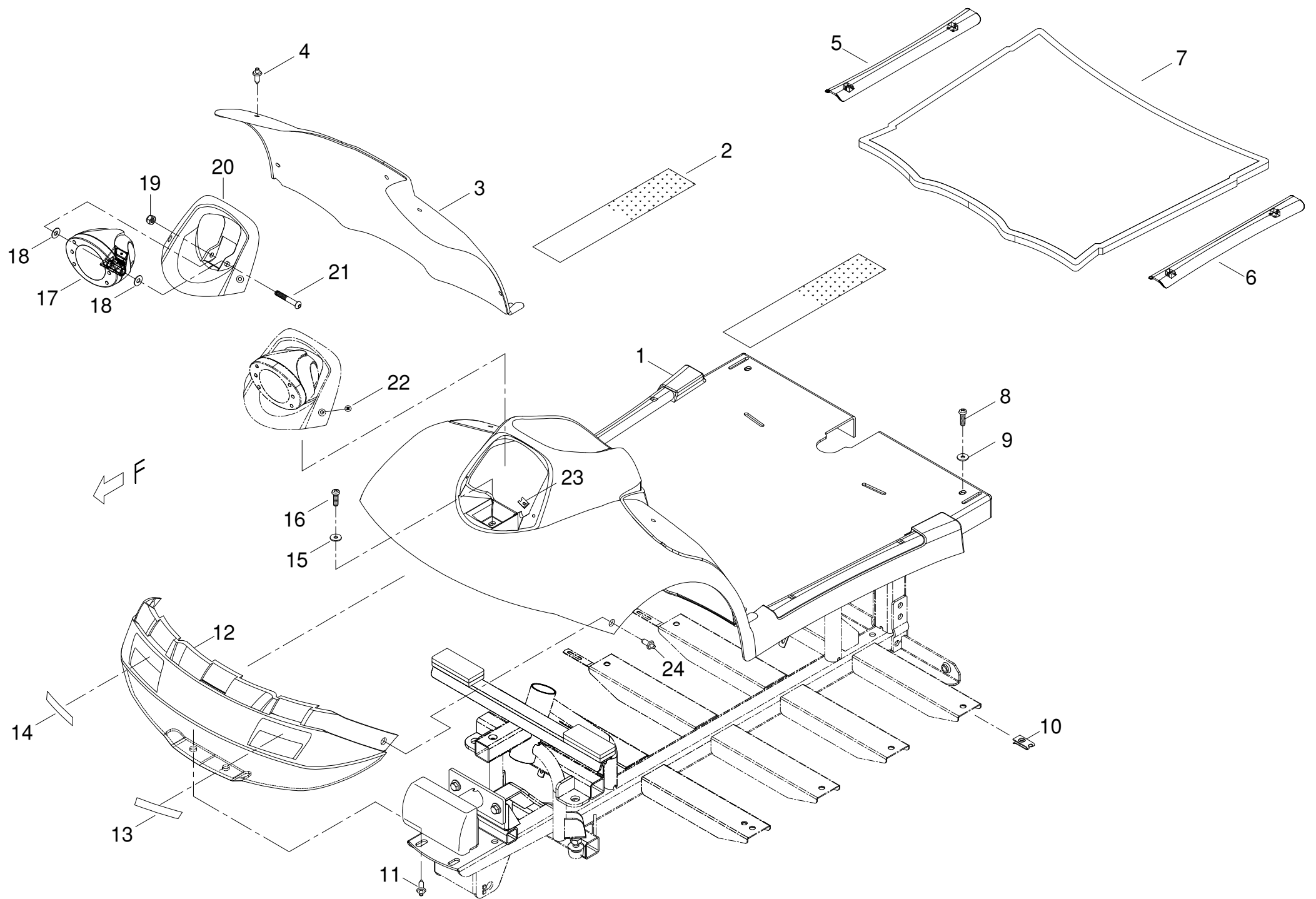


Index NO.	Part NO.	Qty.	Description	Specification	Remark
7-A	103102-88119-22	REF(1)	Front Chassis Cover Assembly	w/ Decal, 888iX, Burgundy Red	
7-1	103101-88102-22	REF(1)	Front Chassis Cover Assembly	888iX, Burgundy Red	
7-2	203103-88100	REF(1)	Velcro	50W*25M,Hook	
7-3	203505-88102	1	Protection Cover for Footrest	888iX, Black Texture	
7-4	209912-88100	5	PUSH PIVET	F24R5A*6X-J046,Black	
			-----*-----		
7-5	203502-67201	1	Protect cover	R,889XLS#1	
7-6	203501-67201	1	Protect cover	L,889XLS #1	
7-7	209906-88103	1	Carpet	888iX	
7-8	5005-06020-440	4	Screw	M6*P1.0*20L,SUS	
7-9	5021-06018-440	4	Flat Washer	φ 6*18*1.6t,SUS	
7-10	202921-88100	2	U type plate nut	Close	
7-11	209912-88100	2	PUSH PIVET	F24R5A*6X-J046,Black	
7-12	103111-88100-50	1	Front Bumper	888iX,Black	
7-13	203106-88100	1	Reflection Sticker	Left, White	
7-14	203106-88101	1	Reflection Sticker	Right, White	
			-----*-----		
7-15	5021-06018-440	1	Flat Washer	φ 6*18*1.6t,SUS	
7-16	5005-06020-440	1	Screw	M6*P1.0*20L,SUS	
			-----*-----		
7-17	108101-00100	1	LED Headlight	24V/3W,KS-200	
7-18	4001068-00	2	Flat Washer	A68, φ 8*13*1.5t	
7-19	5012-06010-702	1	Nylon lock nut	M6*P1.0*10*8t,Black Galvanized	
7-20	103512-88100-50	1	Cover assembly	888iX,Black	

Index NO.	Part NO.	Qty.	Description	Specification	Remark
7-21	5003-06040-720	1	ALLEN BUTTON HEAD SCREW	M6*P1.0*40L,Black Galvanized	
7-22	5009-04019-710	2	Imperial Screw	5/32"*3/4"L,Black Galvanized	
7-23	202921-88101	2	U type plate nut	4mm,A-00007-03	
7-24	209912-88100	2	PUSH PIVET	F24R5A*6X-J046,Black	
-----	301217-05-A3	1	Trademark Decal, SHOPRIDER	T17-5-A3, 78.5*32	
			-----*-----		
7-A	103102-88119-4A	REF(1)	Front Chassis Cover Assembly	w/ Decal, 888iX, Blue	
7-1	103101-88102-4A	REF(1)	Front Chassis Cover Assembly	888iX, Blue	
			-----*-----		
7-A	103102-88119-67	REF(1)	Front Chassis Cover Assembly	w/ Decal, 888iX, Champagne Silver	
7-1	103101-88102-67	REF(1)	Front Chassis Cover Assembly	888iX, Champagne Silver	
			-----*-----		

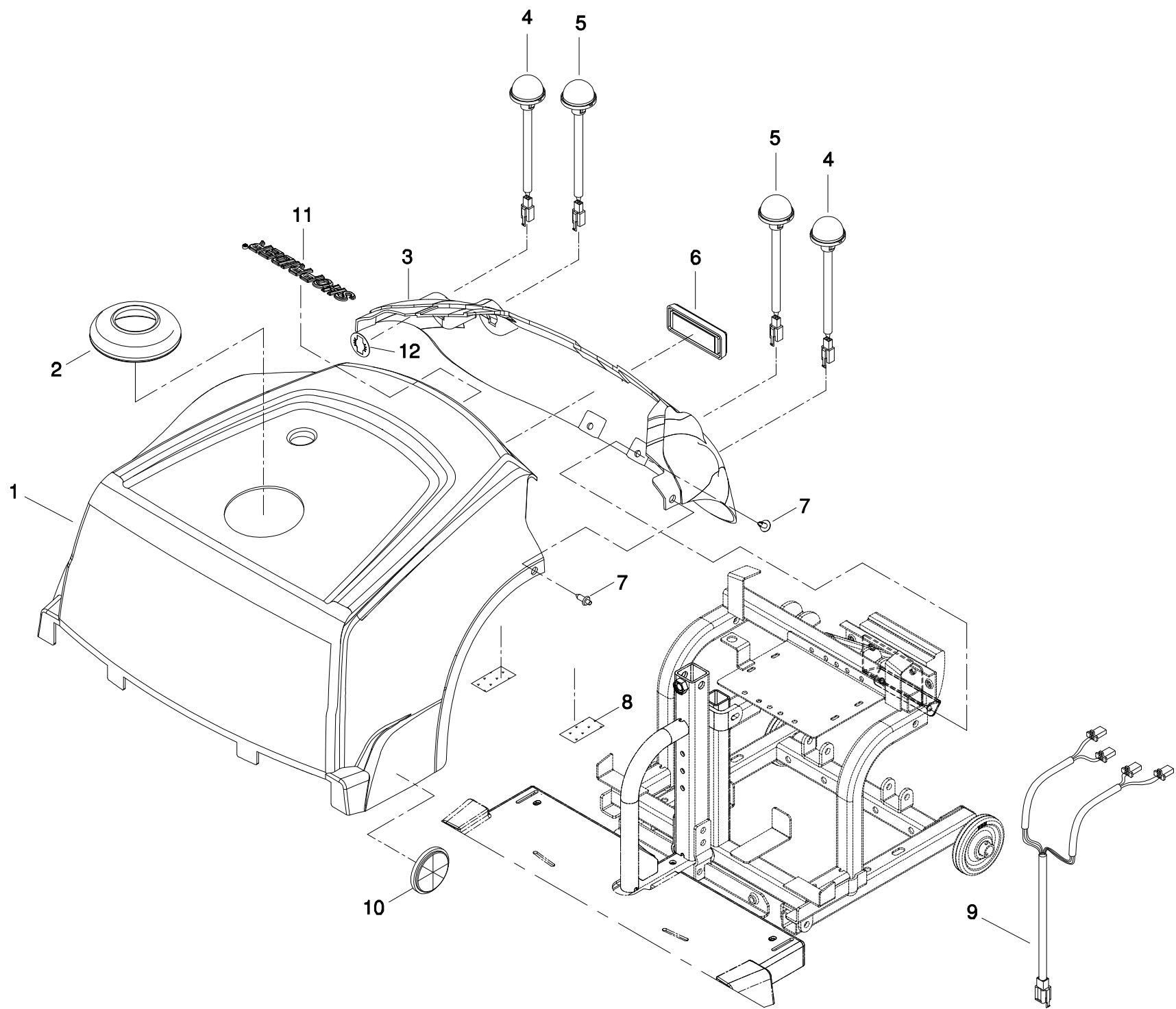


Index NO.	Part NO.	Qty.	Description	Specification	Remark
8-A	103202-88136-22	REF(1)	Rear Chassis Cover Assembly	w/ Decal, 888iX, Burgundy Red	
8-1	103201-88105-22	REF(1)	Rear Chassis Cover Assembly	888iX, Burgundy Red	
8-2	209904-78100	1	Rubber Boot	A7	
8-3	103111-88101-50	1	Rear Bumper	888iX,Black	
8-4	108103-67101-A3	2	Winker/Indicator	Round/Plug, Amber, FUAN 0054, A3	
8-5	108102-67101	2	Tail Light Assembly	Round/Plug,Red,FUAN 0054	
8-6	203205-88102	1	Reflector	E4, JUTE 9605 (PS50-R-T/A), Red	
8-7	209912-88100	4	PUSH PIVET	F24R5A*6X-J046,Black	
8-8	203103-80100	REF	Velcro	25W*25M,Loop	
8-9	207018-67200	1	Winker Cable	RA-889-N	
8-10	203205-88100-A2	2	Reflector	TF-301Y,Yellow(E1)	
8-11	209939-68500	1	Trademark	SHOPRIDER	
8-12	4001453-00	4	Snap Fastener for Lights	A453	
-----	203103-66500	REF	Velcro	25W*25M,Hook	
-----	203103-66400	REF	Velcro	50W*25M,Loop	
-----	301174-69	1	Warning Sticker	N74-69	
-----	301116-01	1	MIT Sticker	N16-1	
-----	301153-18	1	Battery Wiring Diagram Sticker	N53-18, 30A/10A	
-----	301187-11	1	ON-BRAKE-OFF Label	N87-11	
-----	301166-02	1	CE Mark Label	N66-2, White Sign	

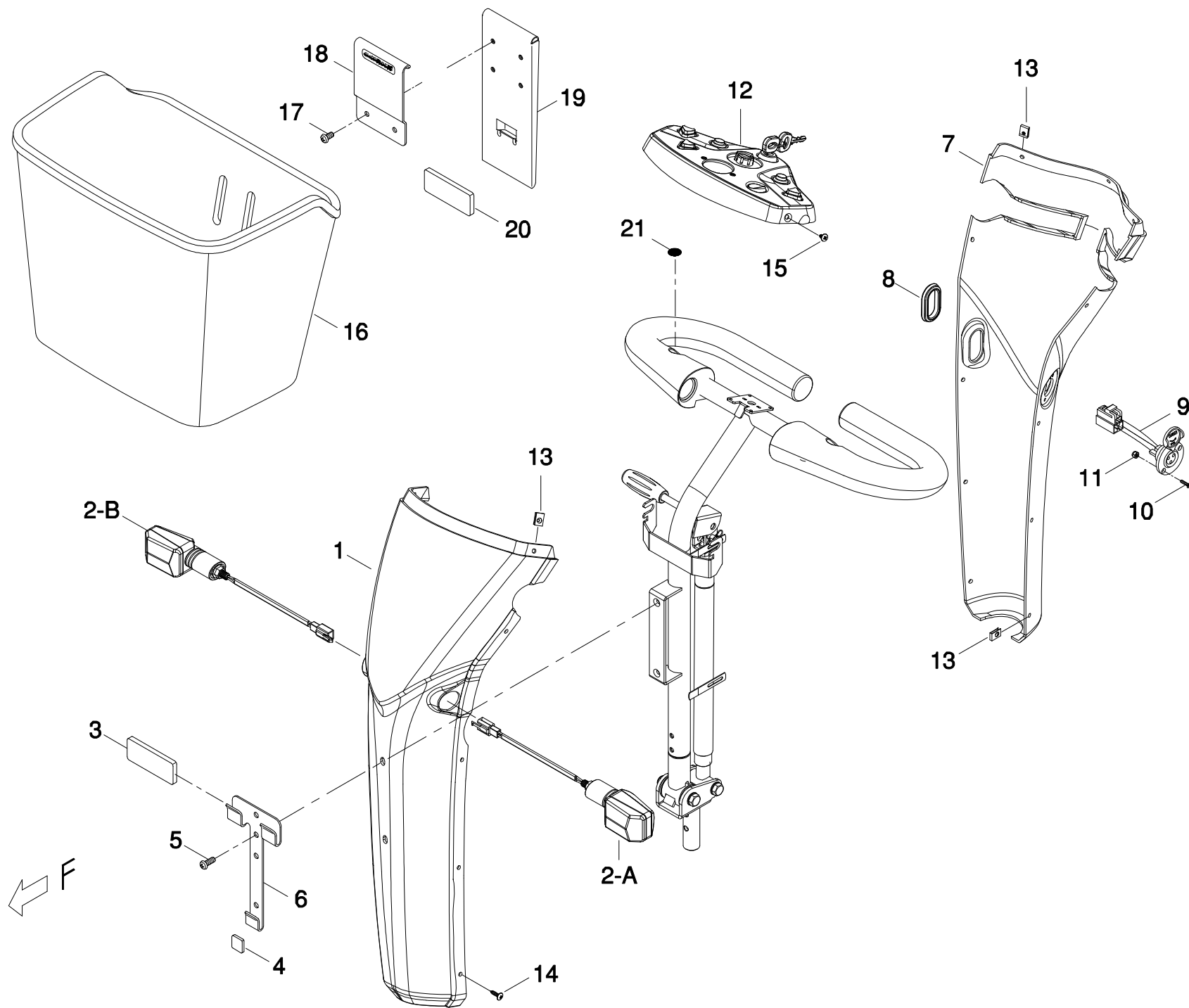


Index NO.	Part NO.	Qty.	Description	Specification	Remark
9	-----		Handle bar cover assembly		
9-1	103301-88107-22	REF(1)	Front Tiller Cover	888iX, Burgundy Red	
9-2-A	108103-88108	1	Winker/Indicator Light	Lift, 888iX	
9-2-B	108103-88109	1	Winker/Indicator Light	Right,888iX	
9-3	301307-00	1	Sponge Sticker	75*30*6t	
9-4	301309-02	1	Sponge Sticker	20*15*3t	
9-5	5005-06016-440	2	Philip Cap Head Bolt	M6*P1.0*16L,SUS	
9-6	4001151-02	1	Basket Holder	A151-2	
9-7	203401-88100	1	Rear Tiller Cover	888iX, Black Texture	
9-8	209931-89100-A2	1	Rubber Plug	for Gas Lift Kit, A2	
9-9	207006-88109	1	Charging cable	KA-888-H11	
9-10	5009-03010-440	2	Taper Head Philip Screw	M3*P0.5*10L,SUS	
9-11	5012-03006-200	2	Nyloc Nut	M3*P0.5*6,Multicolored Galvanized	
9-12	-----	REF(1)	Front Tiller Cover Assembly	888iX, Silver	
9-13	202921-88101	13	U type plate nut	4mm,A-00007-03	
9-14	5006-04016-710	8	Self tapping screw	M4*16L,Black Galvanized	
9-15	5009-04016-440	5	Socket Head Screw, \$ 7mm	M4*16L, SUS	
			-----*-----		
9-16	209916-88107	1	Basket	AO-04, Black Plastic	
9-17	5005-06012-440	2	Screw	M6*P1.0*12L,SUS	
9-18	4001357-02-A2	1	Basket Bracket	Front, A357-2-A2, SHOPRIDER	
9-19	4001358-00-A3	1	Mounting Hook for Basket	Rear, A358-A3	
9-20	301307-00	1	Sponge Sticker	75*30*6t	
9-21	301114-07	2	Round sticker	N14-7,PVC	

Index NO.	Part NO.	Qty.	Description	Specification	Remark
-----	9007-002	REF	Silicone Sealant, Black	DCMB(4000403), for 889XLSN(Cabin)	
-----	301123-30-A2	1	Note Sticker	N23-30-A2, for Sleeping Mode	
-----	301123-20	1	Warning sticker	N23-20,Front Panel	
-----	301174-45-A2	1	Warning Sticker	N74-45, A2	
-----	301174-16	1	Warning Sticker	N74-16	
-----	301187-16	1	Instruction sticker	N87-16	
			-----*-----		
9-1	103301-88107-4A	REF(1)	Front Tiller Cover	888iX, Blue	
			-----*-----		
9-1	103301-88107-67	REF(1)	Front Tiller Cover	888iX, Champagne Silver	

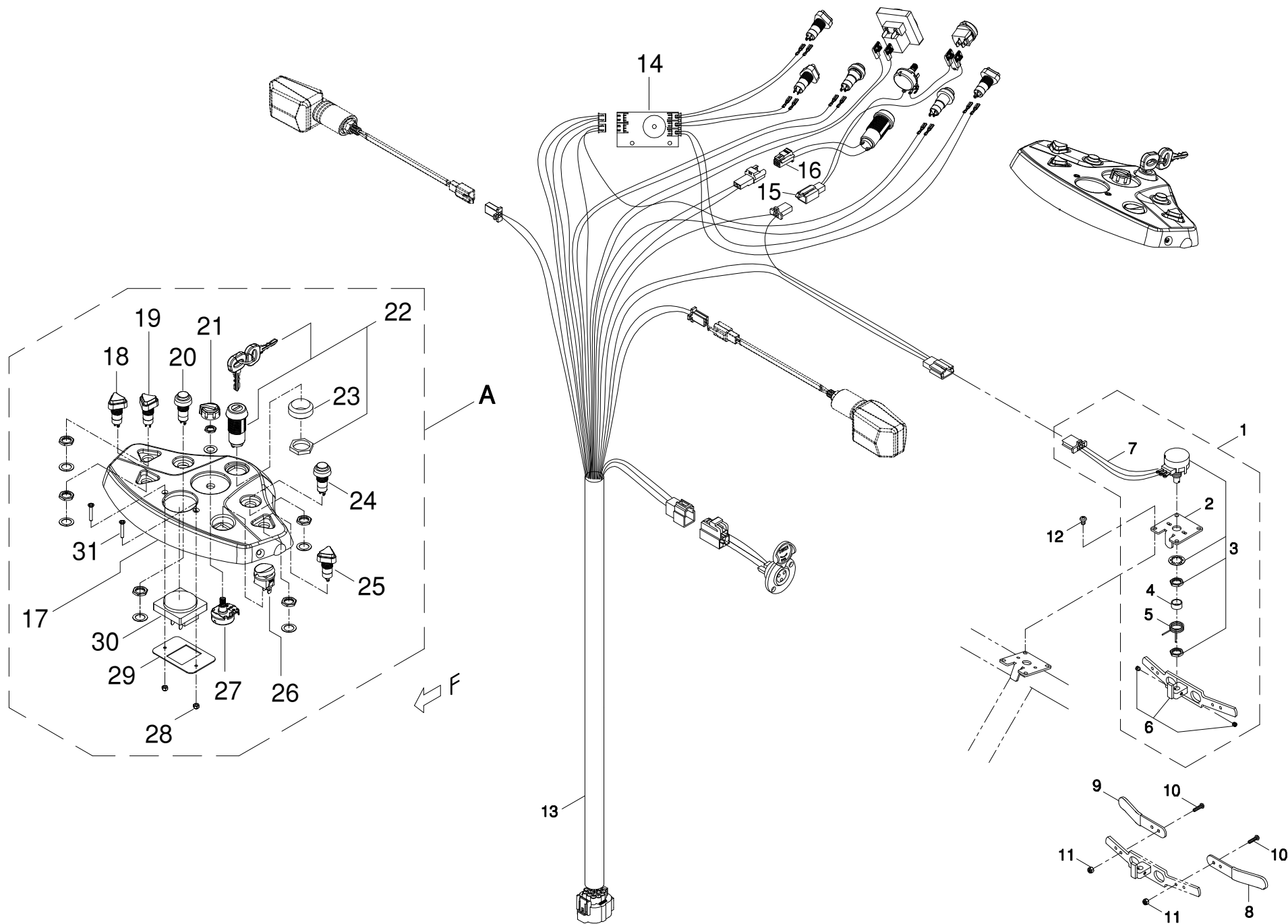


Index NO.	Part NO.	Qty.	Description	Specification	Remark
10	-----		Upper controller assembly		
10-1	106108-66403-A2	1	Potentiometer	T-bar,5K#3	
10-2	4001144-03	1	Potentiometer Fixing plate	A144-3, \$ 4.1	
10-3	206103-88103-A2	1	Potentiometer	5K#3-A2	
10-4	4003198-00	1	Spacer	C198(\$ 11.3* \$ 9.5*5.5L)	
10-5	206106-88100	1	Spring		
10-6	206107-88104	1	Throttle lever	#5	
10-7	207010-66402	1	Control cable	KA-889-K6	
10-8	206122-88100	1	Extension, thottle lever	Left,,Black	
10-9	206123-88100	1	Extension, thottle lever	PVC, Yellow#2	
10-10	5005-03010-210	4	Philip Screw	M3*P0.5*10L, Multicolor Galvanized	
10-11	5012-03006-200	4	Nyloc Nut	M3*P0.5*6,Multicolored Galvanized	
10-12	5005-04006-210	2	Screw	M4*P0.7*6L,Multicolored Galvanized	
			-----*-----		
10-13	207010-67505	1	Upper Control Cable	NA-889-K3	
10-14	206204-88102	1	PC Board	PH-34	
10-15	207014-88110	1	Cannection Cable w/ Cement Resistor	KA-888-L13,888iX	
10-16	207020-88102	1	Wire harness, circuit braker	KA-888-R4	
			-----*-----		
10-A	103302-88102-12	1	Front Tiller Cover Assembly	888iX, Silver	
10-17	103301-88106-12	1	Front Tiller Cover	888iX, Silver	
10-18	208202-88103-A2	1	Hazard Warning Switch	2pin,Red	
10-19	208202-88104	1	Hazard Warning Switch	Yellow,PS-200B-G	
10-20	208201-88100-A2	1	Light Switch	Blue	


Index NO.	Part NO.	Qty.	Description	Specification	Remark
10-21	206113-88101-A2	1	Speed Dial Knob	CF-X4-T,A2	
10-22	106106-88103-A3-1	1	Ignition Key Assembly, Yellow	2-Way Key, 4A 125VAC 2Amp 8212, A3-1	
10-23	4003260-00	1	Spacer	C260(\$ 26* \$ 20*12L)	
10-24	206105-88100-A2	1	Horn Switch	Plain Yellow Button	
10-25	208202-88104	1	Hazard Warning Switch	Yellow,PS-200B-G	
10-26	208203-88106	1	Switch Button	RS-14-2	
10-27	206109-77700-A2	1	Variable resistor	B20K(A2)	
10-28	5012-03006-200	2	Nyloc Nut	M3*P0.5*6,Multicolored Galvanized	
10-29	4001542-00	1	Bracket	A542	
10-30	206130-88100-A4	1	Battery Gauge	LED, A4	
10-31	5009-03020-440	2	Screw	M3*P0.5*20L,SUS	
-----	9004-001	REF	Heat shrink sleeve	F32,3*0.25*200M	
-----	9003-011	REF	Insulating Tape	145, 25W, 25M, Black	
-----	9003-009	REF	Insulation Tape	145 50W	
-----	301111-02	1	Instruction sticker	N11-2,L,SPEED,H	
-----	301222-00-A2	1	Decal	T22	
-----	301111-14	1	Instruction sticker	N11-14	

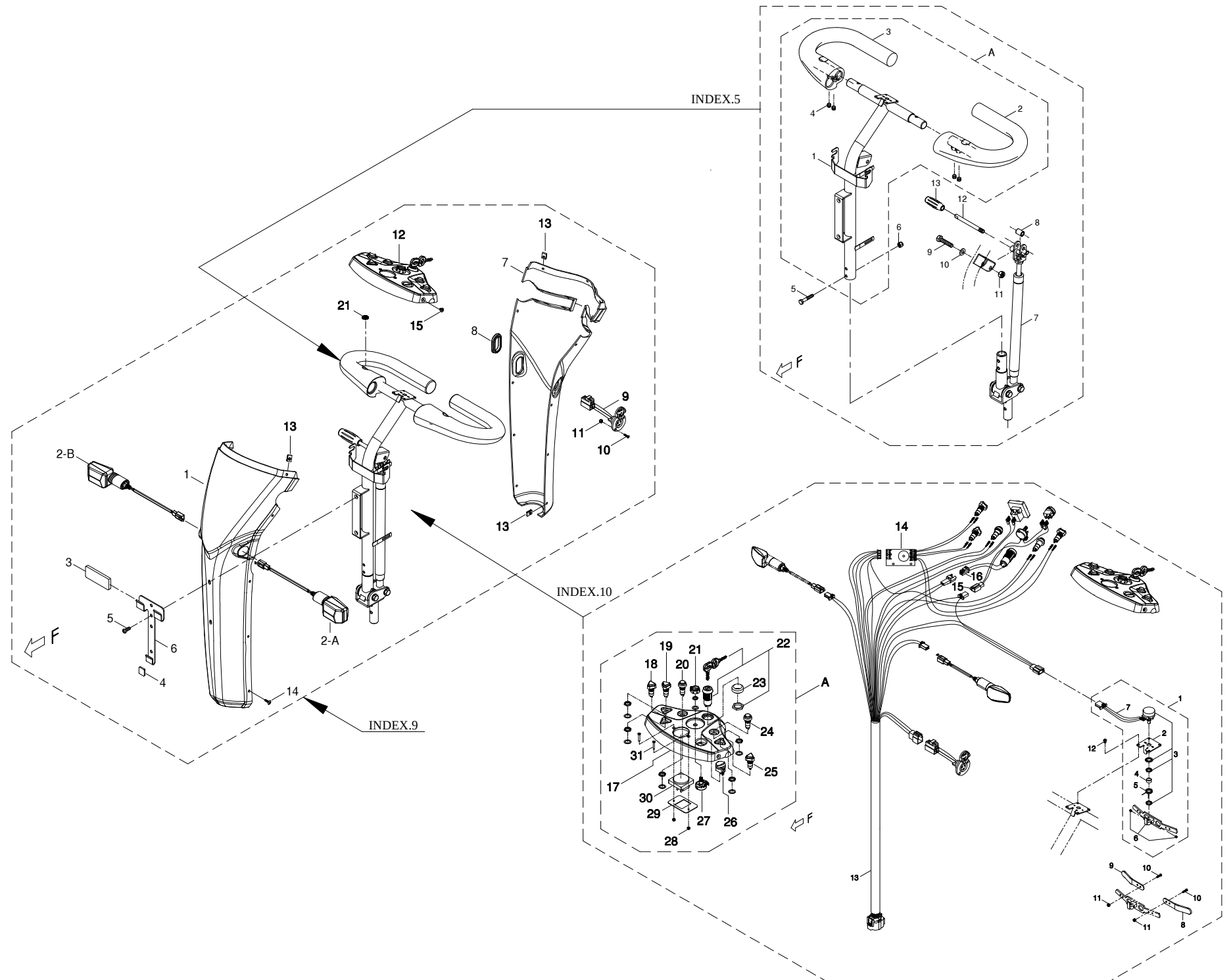


Index NO.	Part NO.	Qty.	Description	Specification	Remark
11	103601-88109-22	REF(1)	P-Handle Bar Assembly, w/ Gas Tiller	888iX, Burgundy Red	
5	-----	REF	Handle Bar, w/ PU Coating Hand Grip	888iX, P-type Tiller	
9	-----	REF	Handle bar cover assembly		
10	-----	REF	Upper Control Assembly		
			-----*-----		
11	103601-88109-4A	REF(1)	P-Handle Bar Assembly, w/ Gas Tiller	888iX, Blue	
			-----*-----		
11	103601-88109-67	REF(1)	P-Handle Bar Assembly, w/ Gas Tiller	888iX, Champagne Silver	
			-----*-----		

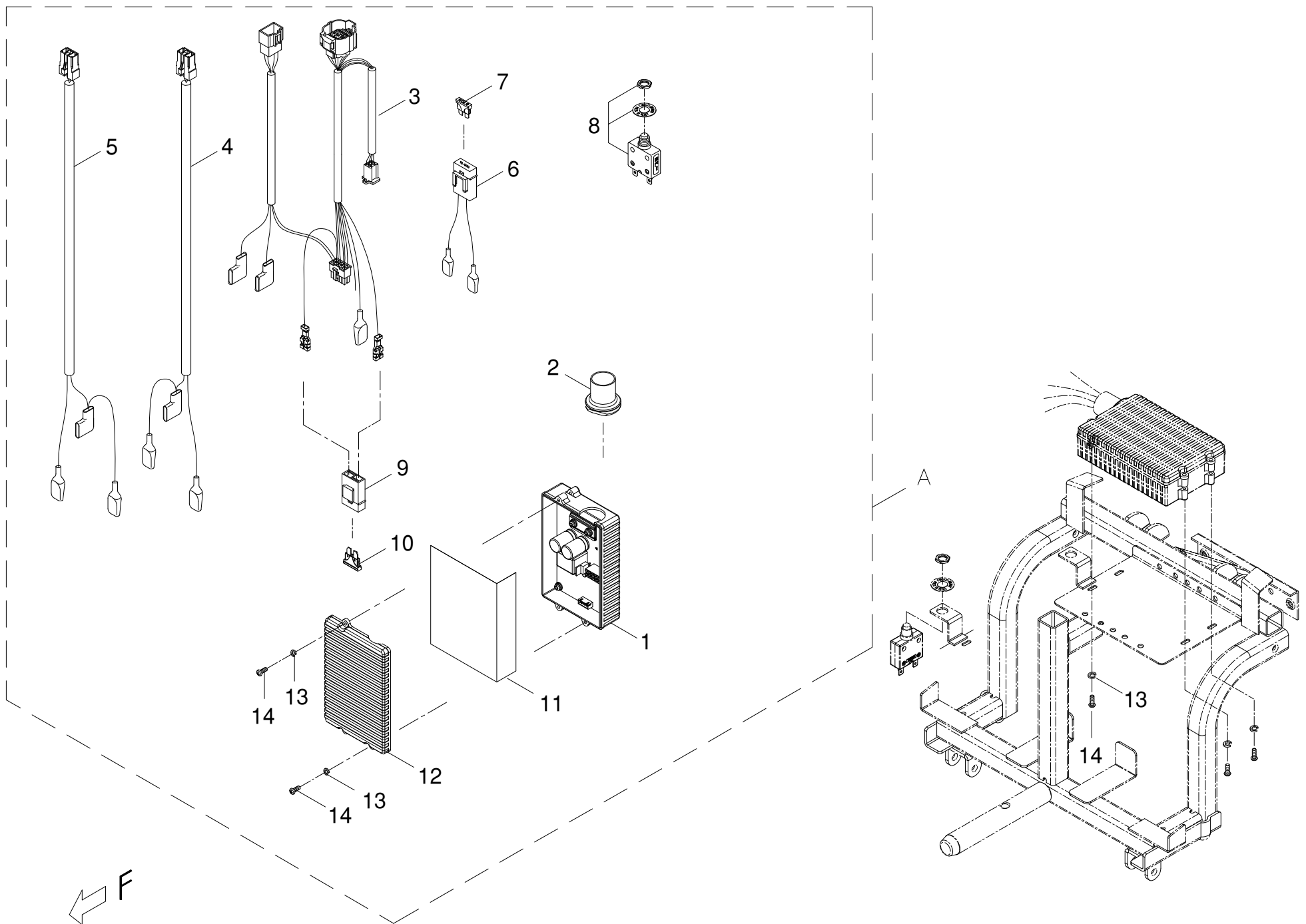

Index NO.	Part NO.	Qty.	Description	Specification	Remark
12-A	106202-88409-A2-1	1	Main Control Box Assembly	AC2-70A, for 889N	
12-1	206221-88201-A3	1	Controller	AC2-70A, w/ Case, A3	
12-2	206210-88100	1	Protection		
12-3	207005-80507	1	Main Control Cable	NA-888-J11	
12-4	207003-89101	1	Power Cable	+,KA-889-D1	
12-5	207004-89101	1	Power Cable	-,KA-889-E1	
12-6	207014-88105	1	Connection Cable	KA-888-L8	
12-7	206901-88111	1	Fuse	UL 10A	
12-8	206207-88100	1	Circuit breaker	30A	
12-9	206206-88101	1	Fuse housing		
12-10	206901-88100	1	Fuse	2A	
12-11	302002-01	1	Insulator	127*129mm*0.25t	
12-12	206209-77200-A2	1	Control box cover	UL8A/B,A2	
12-13	5022-04007-210	5	Spring Washer	§ 4*7*1.25t, Multicolored Galvanized	
12-14	5005-04010-210	5	Philip Screw	M4*P0.7*10L, Multicolor Galvanized	
-----	505201-01500-50	2	Cable Tie	CV-150B	
-----	505201-00940-50	4	Cable Tie	CV-094B	

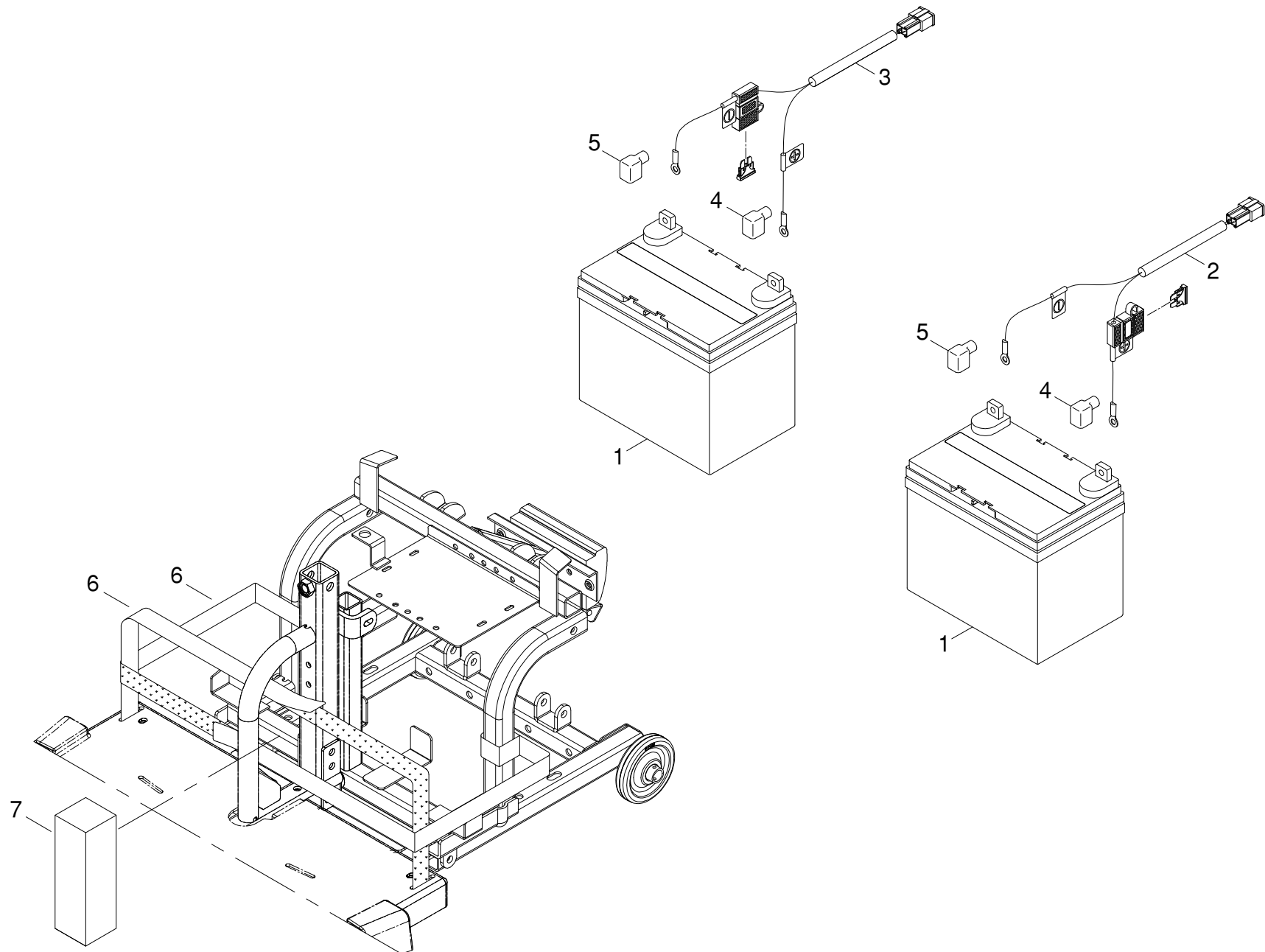

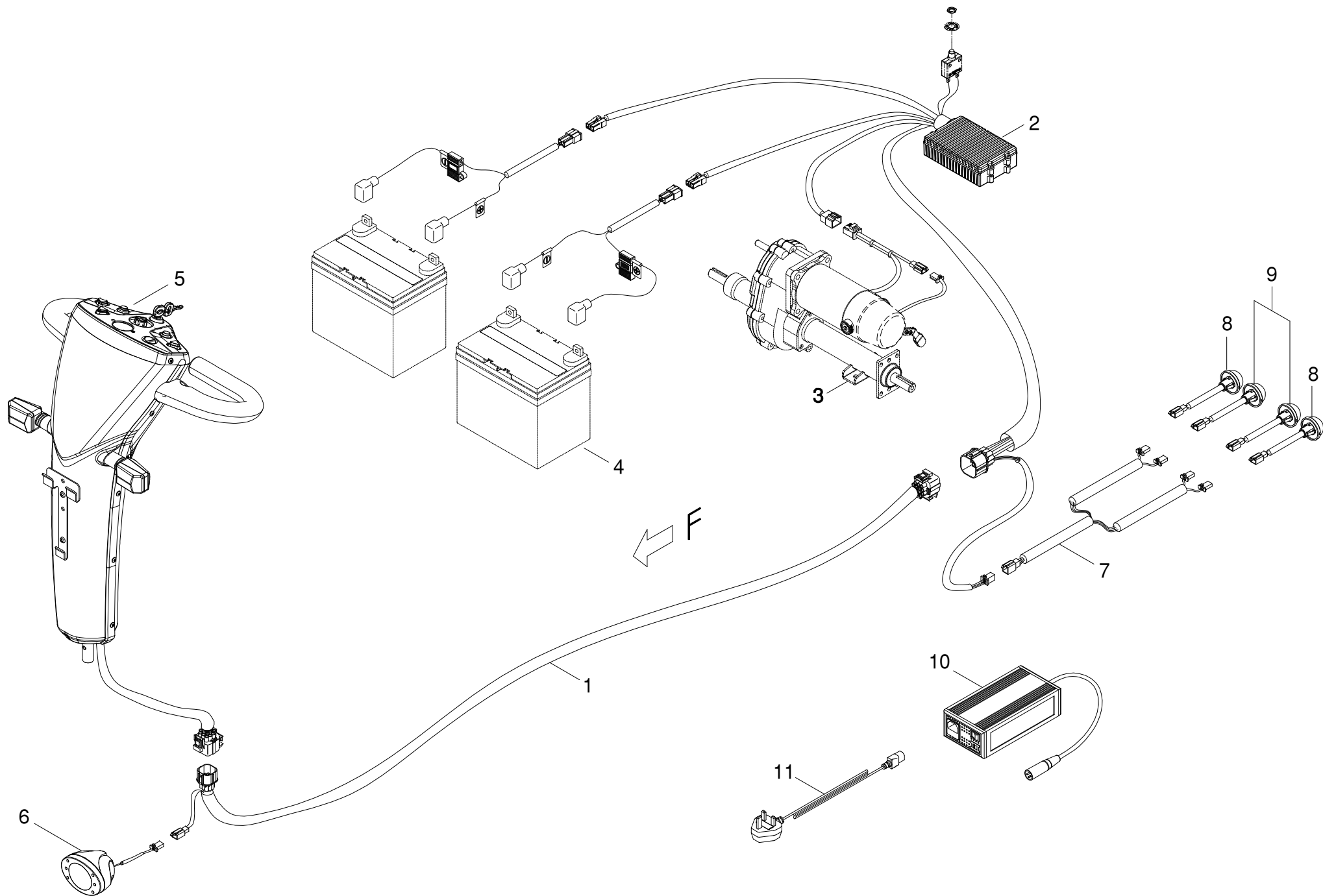

Index NO.	Part NO.	Qty.	Description	Specification	Remark
14	-----		Control and battery wiring illustration		
14-1	207027-88106	1	Control Connection Cable	NA-888-W1	
			-----*-----		
14-2	-----	REF(1)	Main Control Box Assembly	AC2-70A, for 889N	
			-----*-----		
14-3	-----	REF(1)	Drive-train Assembly	8MNF-3, #3, #1-1, A380-24	
			-----*-----		
14-4	-----	REF(2)	SLA Battery w/ Plastic Handle	12V 36Ah,PIHSIANG	
			-----*-----		
14-5	-----	REF(1)	P-Handle Bar Assembly, w/ Gas Tiller	888iX, Burgundy Red	
			-----*-----		
14-6	-----	REF(1)	LED Headlight	24V/3W,KS-200	
			-----*-----		
14-7	-----	REF(1)	Winker Cable	RA-889-N	
14-8	-----	REF(2)	Winker/Indicator	Round/Plug, Amber, FUAN 0054, A3	
14-9	-----	REF(2)	Tail Light Assembly	Round/Plug,Red,FUAN 0054	
			-----*-----		
14-10	109201-88124-A5	1	Off-board Charger, w/ Fan	24V/5A SC1150A-30V, A5, w/ Manual	
			-----*-----		
14-11	209201-88105	1	Power cord	U.K.	

Wiring Diagram for TE-8881A

Legend:
BL: BLUE
BK: BLACK
GR: GREEN
CY: GRAY

Color Synopsis



Components and Connections:

- Top Left:** WINKER, TAIL LIGHT, WINKER. Connects to RA-889-N and WH.
- Top Right:** Fuse 10A, KA-888-L8, AC2-70A.
- Middle Left:** 8MNF-3, KA-888-B1, WH, P/N: 100303-00108, UL7-W-L3, #3L, LED Headlight.
- Middle Right:** PH-34, 20KΩ SPEED POT, V METER, KA-888-L13, 10KΩ 1/4W, SWITCH BUTTON, HORN BUTTON, WINKER BUTTON, KEY SW, KA-888-R4, BL, WH, 5K#3 THROTTLE POT, WH, KA-889-K6.
- Bottom Left:** WINKER, WINKER, WINKER.
- Bottom Center:** KA-888-H11, CHARGER JACK, INHIBIT, BK.
- Bottom Right:** KA-889-D1, RD, KA-889-E1, BK, KA-888-F3A, RD, KA-888-F4A, BK, Fuse 40A, Fuse 40A, BATTERY, BATTERY.

APPENDIX(1)

[back to contents](#)

All Stickers and Decals shown as below and used on TE-888iX

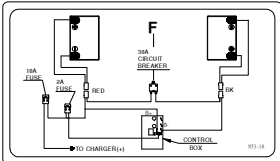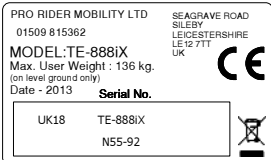 301111-02	 301111-14	 301116-01	 301123-20	 301123-30-A2
 301153-18	 301155-92	 301166-02	 301174-07-A2	 301174-16
 301174-31	 301174-45-A2	 301174-69	 301179-04	 301187-11
 301217-05-A3	 301222-00-A2			