

GMP
ISO9001
ISO14001
ISO13485
CERTIFIED

PIHSIANG®

SHOPRIDER®

MODEL: **TE-9**



(PATENTED)

PIHSIANG MACHINERY MFG. CO., LTD.

NO.108, HSIN-HE ROAD, HSIN-FENG HSIANG 304
HSINCHU COUNTY, TAIWAN


MODEL :

[illegible]


<FRONT PAGE=
TE-9>

TABLE OF CONTENTS ILLUSTRATION

INDEX 1	FRONT CHASSIS ASSEMBLY -----	前車架總成 -----	1-1~2
INDEX 2	REAR CHASSIS ASSEMBLY -----	後車架總成 -----	2-1~3
INDEX 3	DRIVE-TRAIN ASSEMBLY -----	馬達剎車變速箱總成 -----	3-1
INDEX 4	REAR WHEEL ASSEMBLY -----	後輪總成 -----	4-1
INDEX 5	FRONT AXLE ASSEMBLY -----	導向輪總成 -----	5-1~3
INDEX 6	CONNECTING ROD ASSEMBLY -----	導向連桿總成-----	6-1
INDEX 7	FRONT WHEEL ASSEMBLY -----	前輪總成-----	7-1
INDEX 8	HANDLE BAR W/HAND GRIP -----	P型把手+披覆握把套加工/配合氣壓棒 -	8-1
INDEX 9	DELUXE SEAT ASSEMBLY -----	大折疊椅模組+滑道 -----	9-1
INDEX 10	FRONT CHASSIS COVER ASSEMBLY -----	前蓋板總成 -----	10-1
INDEX 11	REAR CHASSIS COVER ASSEMBLY -----	後蓋板總成 -----	11-1
INDEX 12	HANDLE BAR COVER ASSEMBLY -----	把手蓋總成 -----	12-1~2
INDEX 13	TILLER REAR COVER ASSEMBLY -----	把手下蓋總成(含喇叭)-----	13-1
INDEX 14	UPPER CONTROL ASSEMBLY -----	上控制總成 -----	14-1~2
INDEX 15	HANDLE BAR ASSEMBLY -----	把手總成(大)-----	15-1
INDEX 16	MAIN CONTROL BOX ASS'Y(W/ BRACKET ETC.) --	下控制總成(P把)含固定片配件 --	16-1
INDEX 17	BATTERY CABLES -----	加電瓶模組-----	17-1
INDEX 18	CONTROL AND BATTERY WIRING ILLUSTRATION ---	控制及電瓶導線總成-----	18-1
	APPENDIX(1) -----	附錄(1)-----	

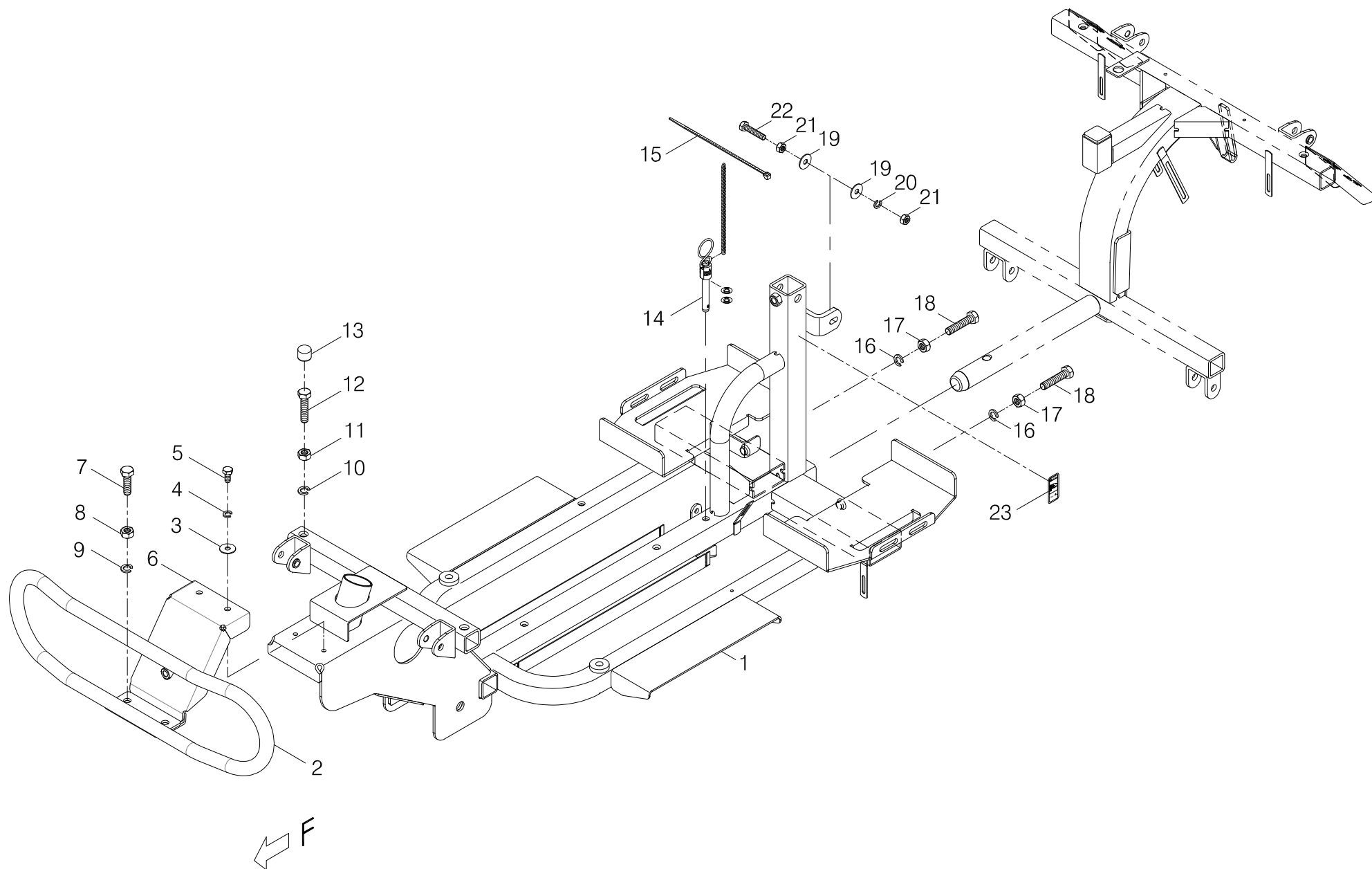


Index NO.	Part NO.	Qty.	Description	Specification	Remark
1	-----		Front chassis assembly		前車架總成
1-1	102101-67102	1	Front chassis assembly	888SL,889SL-13"Tire	前車架總成/888SL,889SL-13"胎
			-----*-----		
1-2	202901-70301	1	Front Bumper	TE-9	前保險桿/AU01-TE-9
1-3	5021-06018-210	2	Flat Washer	φ 6*18*1t,Multicolored Galvanized	平墊圈/φ 6*18*1t (五彩)
1-4	5022-06010-210	2	Spring Washer	φ 6*10,Multicolored Galvanized	彈簧墊圈/φ 6*10 (五彩)
1-5	5001-06012-210	2	Hex head screw	M6*P1.0*12L,Multicolored Galvanized	外六角螺栓/M6*P1.0*12L (五彩)
1-6	202907-70300	1	Bracket for Front Chassis Cover	TE-9	前蓋支撐架/TE-9
1-7	5001-08020-720	2	Hex Head Bolt	M8*P1.25*20L, Black Galvanized	外六角鋼栓(8.8級)/M8*P1.25*20L(鍍黑鋅)
1-8	5011-08013-200	2	Hex Nut	M8*P1.25*13,Multicolored Galvanized	六角螺帽/M8*P1.25*13邊 (五彩)
1-9	5022-08013-210	2	Spring washer	φ 8*13,Multicolored Galvanized	彈簧墊圈/φ 8*13 (五彩)
			-----*-----		
1-10	5022-08013-210	2	Spring washer	φ 8*13,Multicolored Galvanized	彈簧墊圈/φ 8*13 (五彩)
1-11	5011-08013-200	2	Hex Nut	M8*P1.25*13,Multicolored Galvanized	六角螺帽/M8*P1.25*13邊 (五彩)
1-12	5001-08035-210	2	Hex head screw	M8*P1.25*35L,Multicolored Galvanized	外六角螺栓/M8*P1.25*35L (五彩)
1-13	209929-77401	2	Rubber Cover	Black, φ 12*12L	膠套(黑)/φ 12*12L(UL7-WR)
			-----*-----		
1-14	202922-00103	1	Safety Pin	φ 7.93,With two loops	安全銷(含鍊條12L)/φ 7.93(二環)
1-15	505201-01500-50	1	Cable tie	CV-150B	束帶/CV-150B
			-----*-----		
1-16	5022-08013-210	2	Spring washer	φ 8*13,Multicolored Galvanized	彈簧墊圈/φ 8*13 (五彩)
1-17	5011-08013-200	2	Hex Nut	M8*P1.25*13,Multicolored Galvanized	六角螺帽/M8*P1.25*13邊 (五彩)
1-18	5001-08040-210	2	Hex head screw	M8*P1.25*40L,Multicolored Galvanized	外六角螺栓/M8*P1.25*40L (五彩)
			-----*-----		

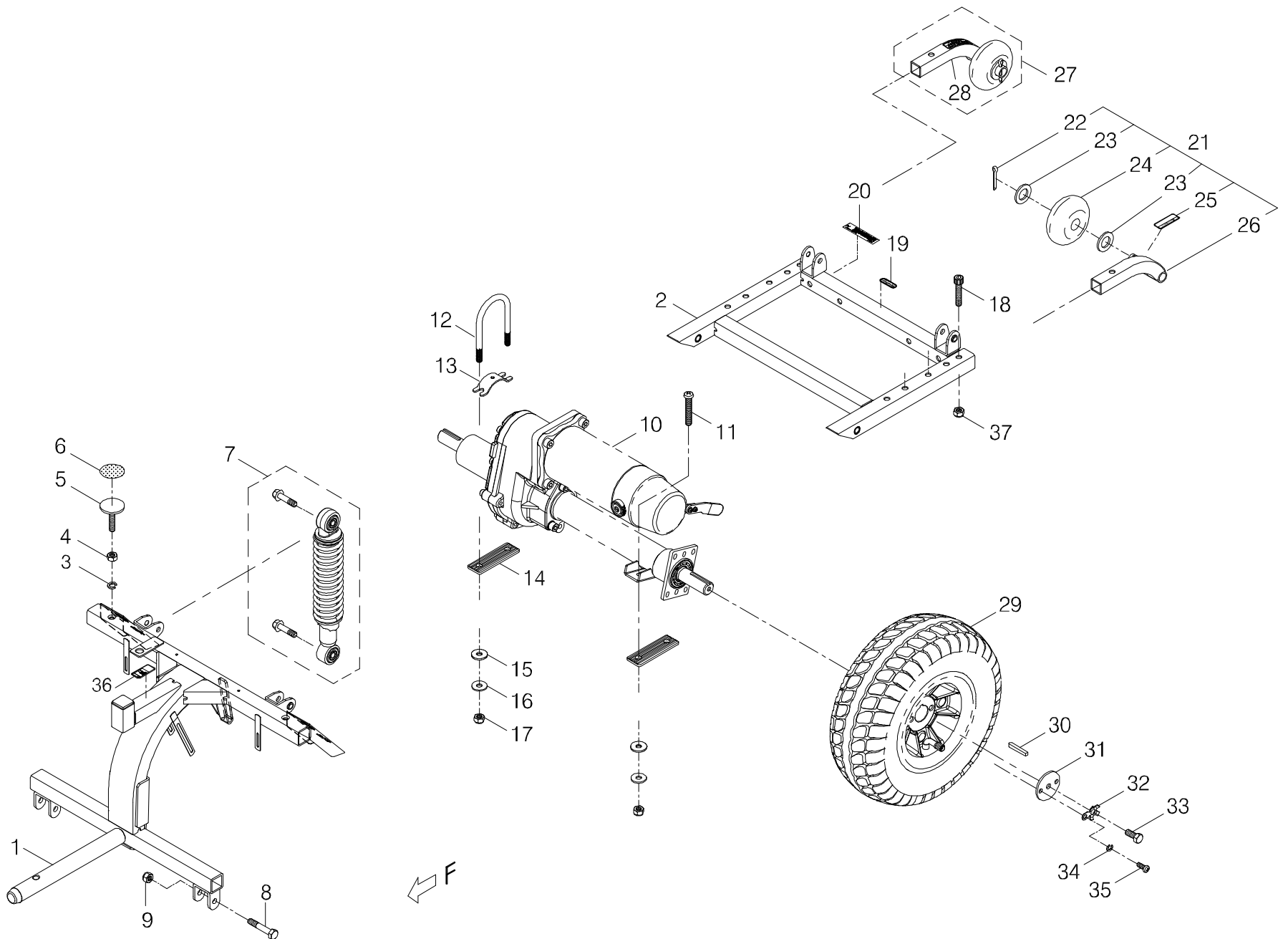


[back to contents](#)


Index NO.	Part NO.	Qty.	Description	Specification	Remark
2	-----		Rear chassis assembly		後車架總成
2-1	102202-67101-A2	1	Rear Chassis Assembly	Upper, 888SL/889SL, A2	後車架(上)總成/888SL/889SL(A2)
2-2	102203-67101-A2	1	Rear Lower Chassis	for 889SEL/SL w/ #1A Transaxle, A2	後車架(下)/889SEL/SL(配合#1A變速箱)(A2)
			-----*-----		
2-3	5022-08013-210	2	Spring washer	φ 8*13,Multicolored Galvanized	彈簧墊圈/φ 8*13 (五彩)
2-4	5011-08013-200	2	Hex Nut	M8*P1.25*13,Multicolored Galvanized	六角螺帽/M8*P1.25*13邊 (五彩)
2-5	2329-00200-01	2	SET , screw	M8*P1.25*35L,Multicolored Galvanized	定位螺栓/ M8*P1.25*35L (五彩)
2-6	2399-00300-00	2	Velcro	φ 35,Hook	自黏扣帶(勾面)/φ 35
			-----*-----		
2-7	202931-89100	2	Adjustable Shock Absorber	w/ Screws, 5-Stage	可調式避震器/含螺栓, 5 段式
			-----*-----		
2-8	5001-08045-720	2	Hex Head Bolt	M8*P1.25*45L,Black Galvanized	外六角鋼栓8.8級/M8*P1.25*45L 半牙(鍍黑鋅)G424
2-9	5012-08013-700	2	Nyloc Nut	M8*P1.25*13,7.8t,Black Galvanized	尼龍螺帽/ M8*P1.25*13邊7.8t(鍍黑鋅)
			-----*-----		
2-10	-----	REF(1)	Drive-Train Assembly		馬達剎車變速箱總成
2-11	5005-08050-710	2	Philip Screw	M8*P1.25*50L,Black Galvanized	十字圓頭螺栓/M8*P1.25*50L (鍍黑鋅)GK7
2-12	4003210-03-A4	1	U-Type Screw, Black Galvanized	C210-3-A4, 787L	U型螺栓(鍍黑鋅)/C210-3-A4, 787L
2-13	4001192-01	1	Arc Plate	A192-1,Black	弧形片/A192-1(黑色)
2-14	209909-69400	2	Rubber Gasket	81*26*4.75t(GK7)	減震膠片(二孔距57mm)/81*26*4.75t(GK7)
2-15	209908-69200C	4	Rubber Washer	787L	變速箱減震墊/ 787L
2-16	5021-08022-710	4	Flat Washer	φ 8*22*2t,Black Galvanized	平墊圈/φ 8*22*2t (鍍黑鋅)
2-17	5012-08013-700	4	Nyloc Nut	M8*P1.25*13,7.8t,Black Galvanized	尼龍螺帽/M8*P1.25*13邊7.8t(鍍黑鋅)
			-----*-----		
2-18	5002-08030-720	2	Cap Head Allen Screw	M8*P1.25*30L, Black Galvanized	圓頭內六角鋼栓/M8*P1.25*30L (鍍黑鋅)

Index NO.	Part NO.	Qty.	Description	Specification	Remark
2-19	301174-09	1	Warning Sticker	N74-9	警告專利貼紙/N74-9
2-20	-----	REF(1)	Decal		條碼貼紙
			-----*-----		
2-21	-----		Anti-tip wheel assembly		防傾輪架(左)總成
2-22	5023-03025-210	1	Split pin	M3*25L,Multicolored galvanized	開口銷/M3*25L (五彩)
2-23	4001305-00	2	Flat Washer	A305, \$ 17.5*30*2.6t	平墊圈/A305 (\$ 17.5*30*2.6t)
2-24	2399-00100-00	1	anti-tip wheel	\$ 17*75*30t	防傾輪/ \$ 17*75*30t
2-25	301174-06	1	Warning sticker	N74-6	警告貼紙/N74-6
2-26	102918-89800	1	Anti-tip wheel frame assembly	Left,889A	防傾輪架(左)總成/889A
			-----*-----		
2-27	-----		Anti-tip wheel assembly		防傾輪架(右)總成
2-22	5023-03025-210	1	Split pin	M3*25L,Multicolored galvanized	開口銷/M3*25L (五彩)
2-23	4001305-00	2	Flat Washer	A305, \$ 17.5*30*2.6t	平墊圈/A305 (\$ 17.5*30*2.6t)
2-24	2399-00100-00	1	anti-tip wheel	\$ 17*75*30t	防傾輪/ \$ 17*75*30t
2-25	301174-06	1	Warning sticker	N74-6	警告貼紙/N74-6
2-28	102919-89800	1	Anti-tip Wheel Frame	Right,889A	防傾輪架(右)總成/889A
			-----*-----		
2-29	-----	REF(1)	Rear Wheel Assembly w/ Brake Hub		後輪總成+剎車轂/灰胎(IA)
2-30	505301-00505-02	2	Drive Key	5*5*35	單頭圓鍵/5*5*35
2-31	4001364-00	2	Flat Washer	A364, \$ 8*45*3t	平墊圈/A364 (\$ 8*45*3t)
2-32	4001481-00	2	Locking tab washer	A481	螺栓防鬆片/A481
2-33	5001-08020-440	2	Hex Head Bolt	M8*P1.25*20L,SUS304	外六角鋼栓(8.8級)/M8*P1.25*20L SUS304
2-34	5022-06010-210	4	Spring Washer	\$ 6*10, Multicolored Galvanized	彈簧墊圈/ \$ 6*10 (五彩)
2-35	5005-06016-443	4	Philip Screw,Nyloc	M6*P1.0*16L,SUS	十字圓頭螺栓/M6*P1.0*16L SUS 耐落

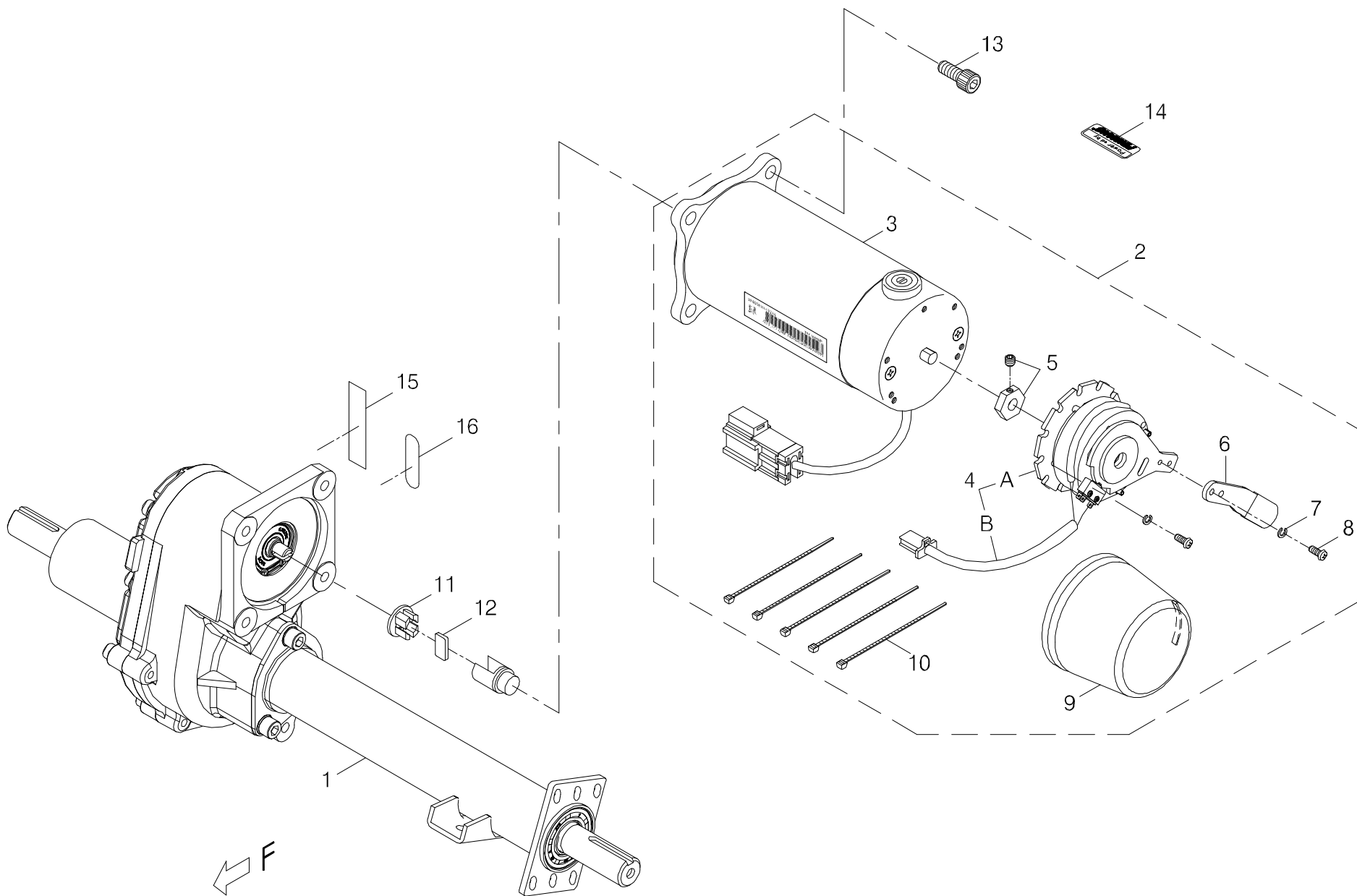


Index NO.	Part NO.	Qty.	Description	Specification	Remark
3	100001-67503-A2	1	Drive-Train Assembly	9MNF-2B(A2), #3, #1A	馬達剎車變速箱總成/9MNF-2B(A2)+(#3)+(#1A)
3-1	100401-88903-A2	1	Transaxle Assembly (#1A)(C)	889SL/SEL, 538TL, A2	變速箱(#1A)(C)/889SL/SEL(538TL)(A2)
3-2	100101-89510-A2	1	Motor and Brake Assembly	9MNF-2B, #3(A380-19)	馬達剎車總成/9MNF-2B+#3(A380-19)
3-3	100201-89608	1	Motor Assembly	9MNF-2B	馬達(總成)/9MNF-2B
3-4	100303-80515-A2	1	Electromagnetic Brake Assembly	#3E-A2,WR229P1,UL7-W-L(A2)	電磁剎車加工總成#3E-A2/WR229P1焊UL7-W-L(A2)
3-A	100306-00100-A4	1	Electromagnetic Brake	#3, WR198A408P1, A4	電磁剎車#3/WR198A408P1(A4)
3-B	2370-00503-00-A2	1	Connection cable	UL7-W-L(A2),385L	連接線/UL7-W-L(385L)(A2)
-----	9004-001	1	Heat shrink sleeve	F32,3*0.25*200M	熱縮套管(不含膠)/F32-3(3*0.25*200M)
-----	9004-010	1	Heat shrink sleeve	F32-5,5*0.25*100M	熱縮套管(不含膠)/F32-5(5*0.25*100M)
3-5	200302-88101-A2	1	Driving Hex Axle for Brake Disc	#2-1, 787N, A2	剎車六角軸套/#2-1 (787N)(A2)
3-6	4001380-19	1	Free wheel extension lever	A380-19(G424SM), Left , Short	離合器加長片/A380-19(G424SM)左，短
3-7	5022-04007-210	5	Spring washer	φ 4*7*1.25t,Multicolored Galvanized	彈簧墊圈/φ 4*7*1.25t (五彩)
3-8	5005-04010-210	5	Philip Screw	M4*P0.7*10L,Multicolored Galvanized	十字圓頭螺栓/M4*P0.7*10L (五彩)
3-9	2399-00201-00	1	Protect Cover for Magnetic Brake	UL7-W/WD	剎車膠套/UL7-W/WD
3-10	505201-00940-50	5	cable Tie	CV-094	束帶/CV-094B
3-11	200904-88100C	1	Joint Adapter		變速箱連接襯套
3-12	200905-88100	1	Plate for Joint Adapter		變速箱連接片
3-13	5002-08020-220	4	CAP HEAD ALLEN SCREW	M8*P1.25*20L,Multicolored Galvanized	圓頭內六角鋼栓12.9級/M8*P1.25*20L (五彩)
3-14	301179-04	1	Trademark Sticker	N79-4	指示貼紙(40*15.5)/N79-4(Powered by PIHSIANG)
3-15	-----	REF(1)	Decal		條碼貼紙
3-16	301174-09	REF(1)	Warning sticker	N74-9	警告專利貼紙/N74-9

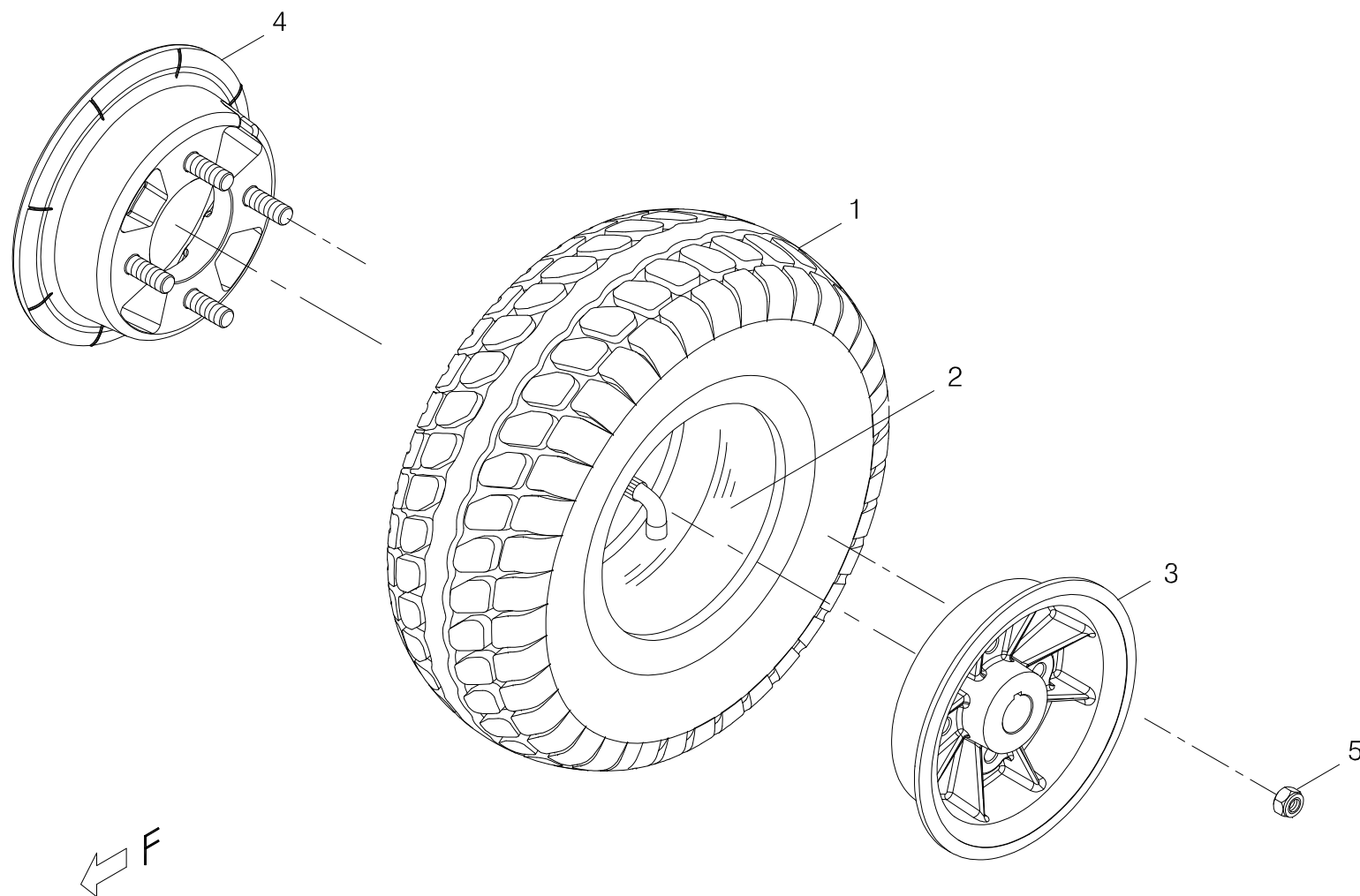


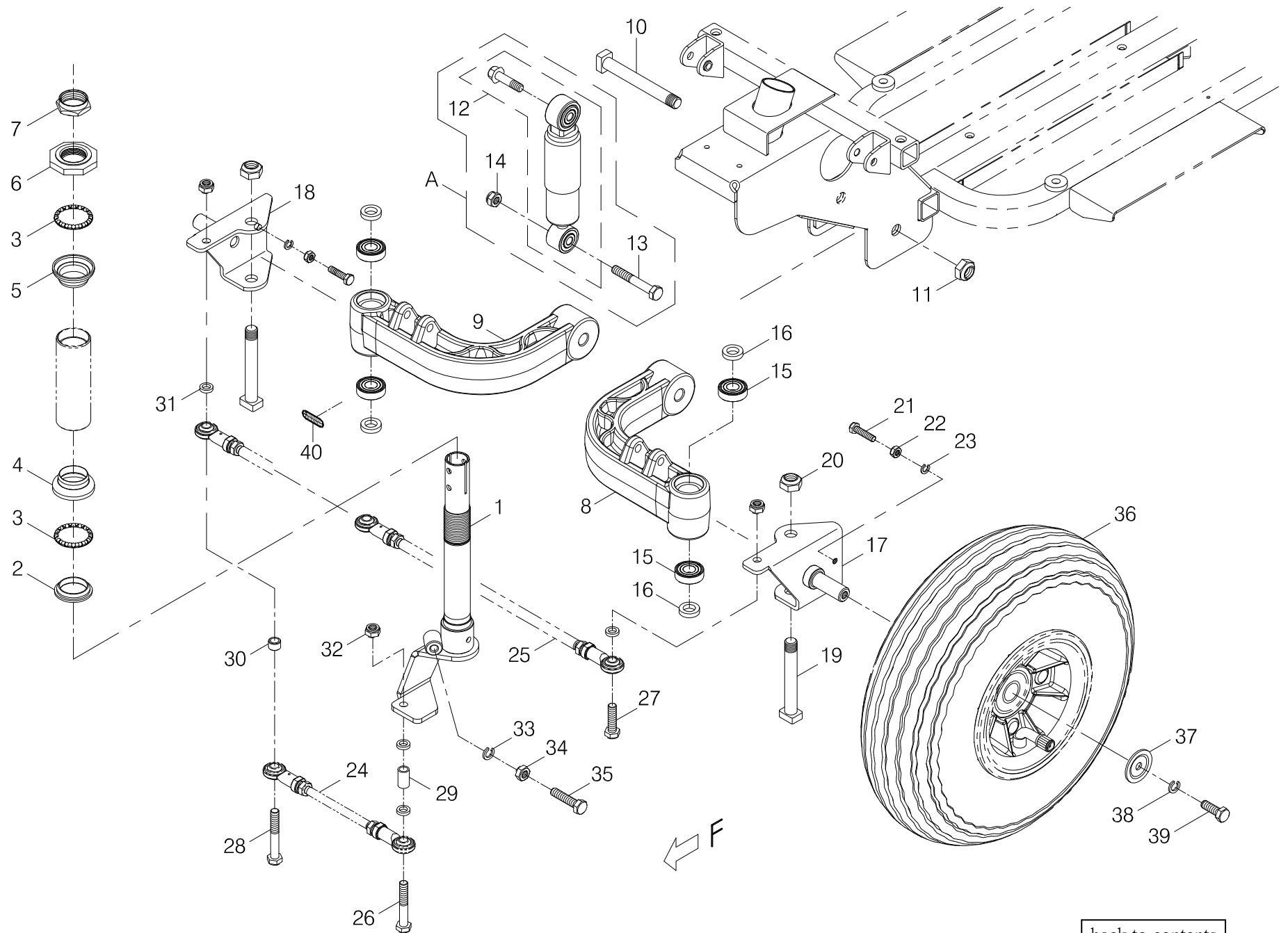


Index NO.	Part NO.	Qty.	Description	Specification	Remark
5	-----		Front axle assembly		導向輪總成
5-1	101501-67100	1	Steering shaft assembly	888SEL,889SEL	操向桿總成/888SEL/889SEL
			-----*		
5-2	201401-88200	1	Bearing bushing	§ 27	軸承座/ § 27
5-3	201402-88200	2	Steering Bearings	5/32*16	軸承/5/32"*16
5-4	201403-88200	1	Bearing housing	§ 30	珠碗(下)/ § 30
5-5	201404-88200	1	Bearing housing	§ 30	珠碗(上)/ § 30
5-6	201405-88200	1	Bearing bushing	Large octangle	軸承襯套/大八角
5-7	201406-88200	1	Steering Head Nut	Dia 22.2	軸承固定螺帽/ § 22.2
			-----*		
5-8	101802-67100-A2C	1	Front Crossbeam	Left, 888SEL/889SEL, #1	前輪橫樑(左)/888SEL/889SEL#1
5-9	101803-67100-A2C	1	Front Crossbeam	Right, 888SEL/889SEL, #1	前輪橫樑(右)/888SEL/889SEL#1
5-10	4003048-07	1	Front fork shaft	C48-7,M12*P1.25*98L,Multicolor	前叉軸/C48-7 (M12*P1.25*98L)五彩
5-11	5014-12019-210	1	Self-lock nut	M12*P1.25*19,Multicolored Galvanized	防鬆螺帽/M12*P1.25*19邊(五彩)
			-----*		
5-A	102930-89501-A2	2	SHOCK ABSORBERS FRONT MX4	889DXN/SEL,888SEL A2	前避震器總成/889DXN/SEL,888SEL(A2)
5-12	202930-89500-A2	1	Front Shock Absorber	889DXN(A2)	前避震器/889DXN(含螺栓)(A2)
5-13	5001-08050-221	1	Hex Head Bolt	M8*P1.25*50L, Multicolored Galvanized	外六角鋼栓/M8*P1.25*50L牙長12L(五彩)
5-14	5012-08013-200	1	Nyloc Nut	M8*P1.25*13 7.8t,Multicolored Galvanized	尼龍螺帽/M8*P1.25*13邊7.8t(五彩)
			-----*		
5-15	505101-60012-01-1	4	Ball Bearing	#6001zz, NBN	滾珠軸承/#6001zz (NBN)(非保稅)
5-16	4005006-00	4	Spacer	H6, § 12ID* § 20OD*4.5T	間隔圈/H6(§ 12 ID* § 20 OD*4.5T)
			-----*		
5-17	202902-66801-A2	1	Front wheel axle	J-8 #1,Left	前輪軸(左)/J-8 #1(A2)


Index NO.	Part NO.	Qty.	Description	Specification	Remark
5-18	202903-66801-A2	1	Front wheel axle	J-8 #1,Right	前輪軸(右)/J-8 #1(A2)
5-19	4003048-06	2	Front fork shaft	C48-6,M12*P1.25*85L,Multicolor	前叉軸/C48-6 (M12*P1.25*85L)五彩
5-20	5014-12019-210	2	Self-lock nut	M12*P1.25*19,Multicolored Galvanized	防鬆螺帽/M12*P1.25*19邊 (五彩)
5-21	5001-06025-220	2	Hex Head Bolt	M6*P1.0*25L,Multicolored Galvanized	外六角鋼栓/M6*P1.0*25L (五彩)8.8級
5-22	5011-06010-200	2	Hex Nut	M6*P1.0*10*5L,Multicolored Galvanized	六角螺帽/M6*P1.0*10邊*5L (五彩)
5-23	5022-06010-210	2	Spring Washer	§ 6*10,Multicolored Galvanized	彈簧墊圈/ § 6*10 (五彩)
			-----*-----		
5-24	-----	REF(1)	Connecting rod assembly (Short)		導向連桿總成(短)
5-25	-----	REF(1)	Connecting rod assembly (Long)		導向連桿總成(長)
5-26	5001-08055-220	1	Hex Head Bolt	M8*P1.25*55L,Multicolored Galvanized	外六角鋼栓/M8*P1.25*55L 半牙(五彩)12.9級
5-27	5001-08030-210	1	Hex head screw	M8*P1.25*30L,Multicolored Galvanized	外六角螺栓/M8*P1.25*30L (五彩)
5-28	5001-08050-220	1	Hex Head Bolt	M8*P1.25*50L,Multicolored Galvanized	外六角鋼栓/M8*P1.25*50L 半牙(五彩)
5-29	4003264-11	1	Spacer	C264-11, § 8*12*19L	間隔圈/C264-11 (§ 8*12*19L)
5-30	4003264-05	1	Spacer	C264-5, § 8.1* § 12*8L	間隔圈/C264-5 (§ 8.1* § 12*8L)
5-31	4001068-01	4	Flat Washer	A68-1, § 8.1*13*3t	平墊圈/A68-1 (§ 8.1*13*3t) 銅
5-32	5012-08013-200	3	Nyloc Nut	M8*P1.25*13 7.8t,Multicolored Galvanized	尼龍螺帽/M8*P1.25*13邊7.8t(五彩)
			-----*-----		
5-33	5022-08013-210	1	Spring washer	§ 8*13,Multicolored Galvanized	彈簧墊圈/ § 8*13 (五彩)
5-34	5011-08013-200	1	Hex Nut	M8*P1.25*13,Multicolored Galvanized	六角螺帽/M8*P1.25*13邊 (五彩)
5-35	5001-08035-210	1	Hex head screw	M8*P1.25*35L,Multicolored Galvanized	外六角螺栓/M8*P1.25*35L (五彩)
			-----*-----		
5-36	-----	REF(2)	Front wheel assembly		前輪總成
5-37	4001329-00	2	Flat Washer	A329	平墊圈/A329
5-38	5022-08013-210	2	Spring washer	§ 8*13,Multicolored Galvanized	彈簧墊圈/ § 8*13 (五彩)

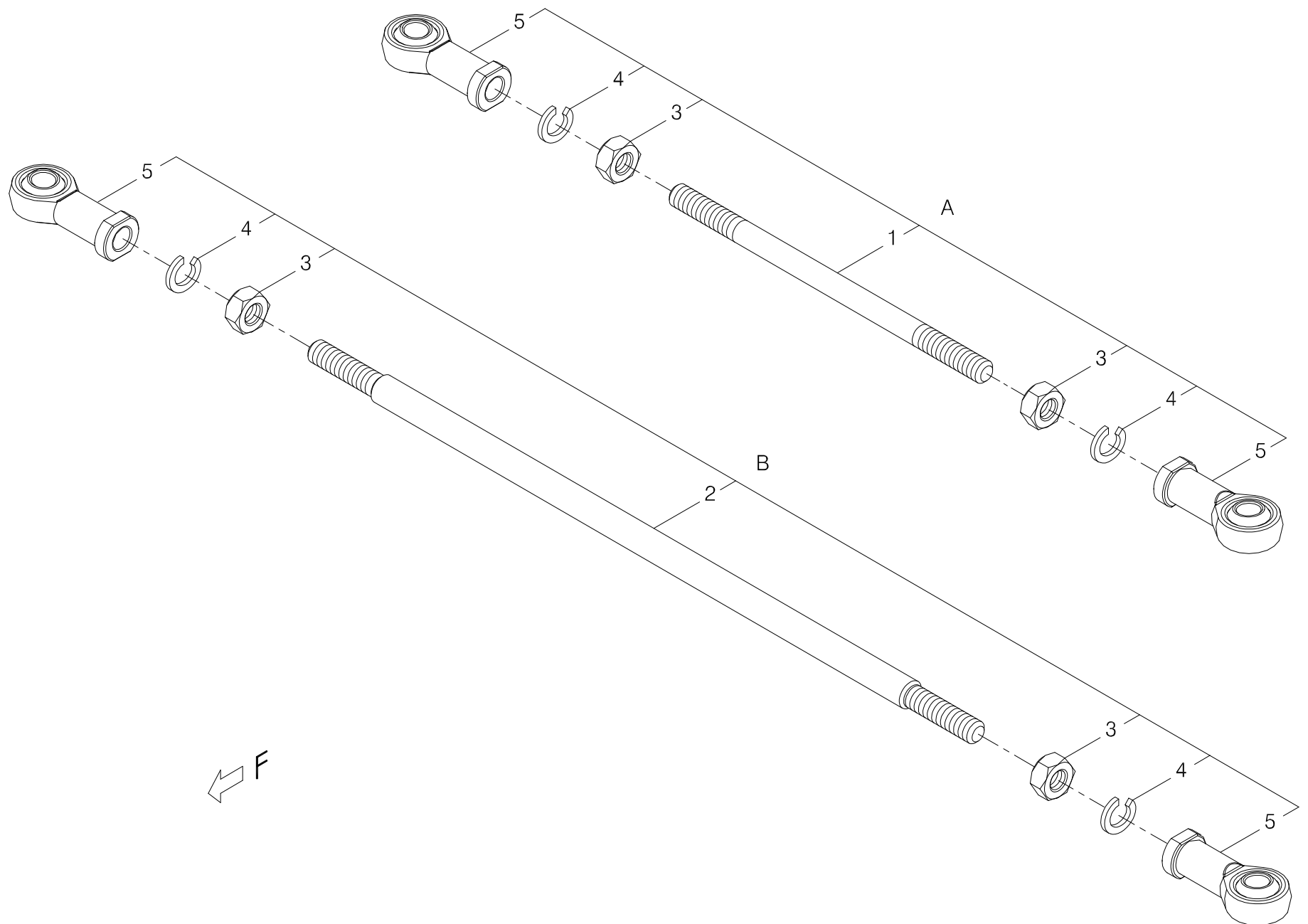

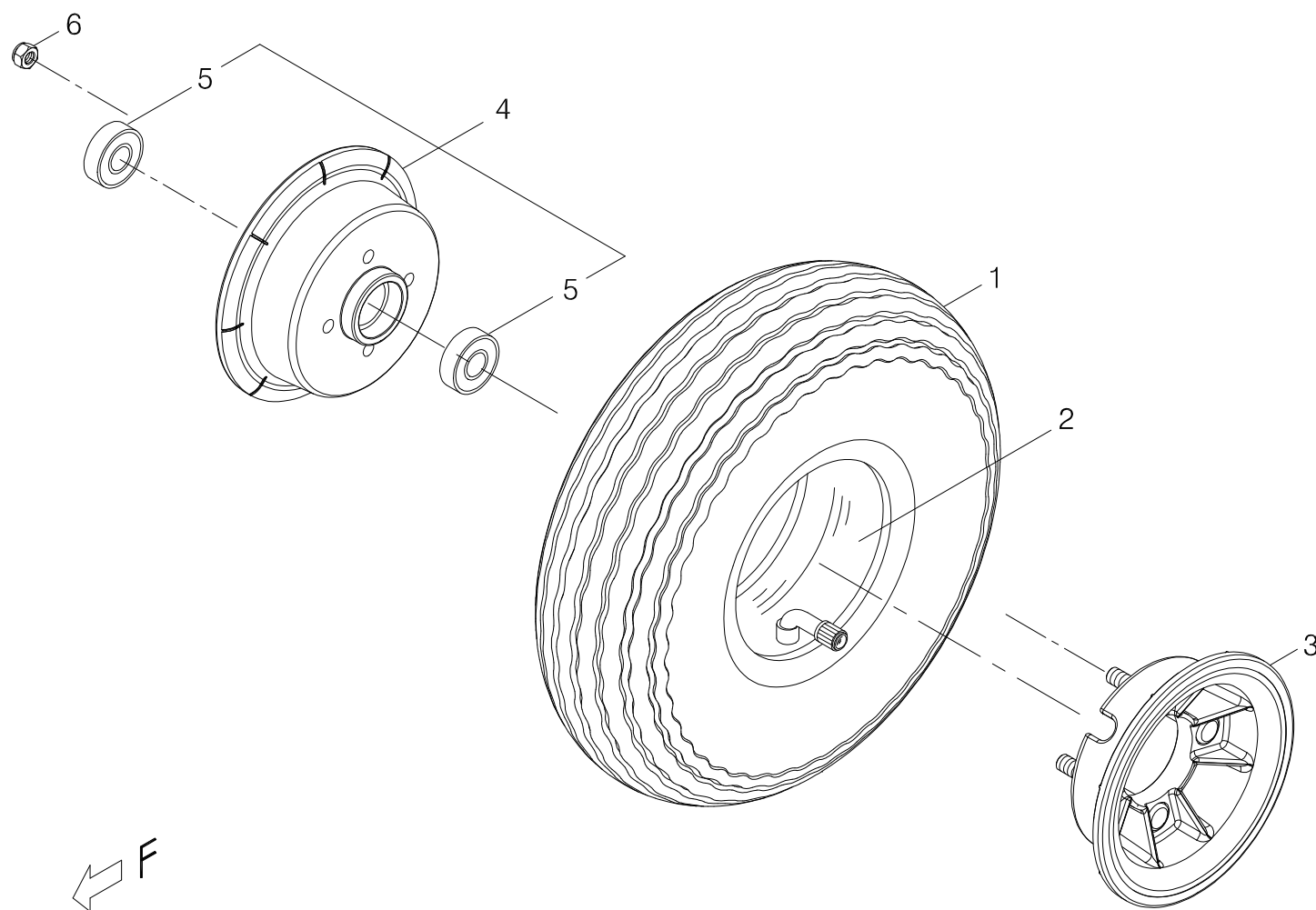

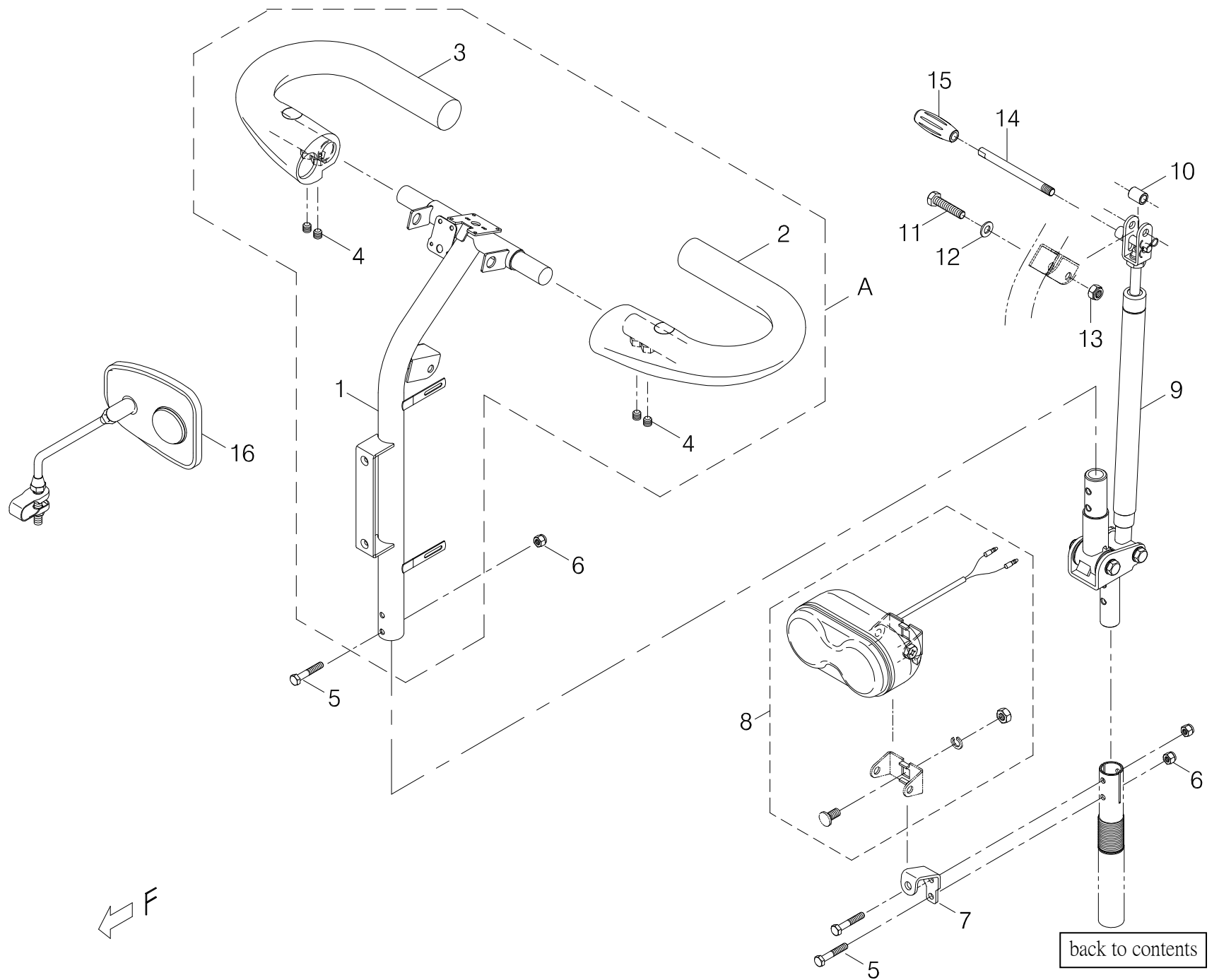


Index NO.	Part NO.	Qty.	Description	Specification	Remark
8-A	101104-89100	1	Handle Bar w/ Hand Grip Assembly	for P-Bar & Gas Lift	P型把手+披覆握把套加工/配合氣壓棒
8-1	101101-89100	1	Loop Handle Frame	P-Bar	把手架/P把
8-2	101906-80501	1	Loop handle bar	Left,888A#1,Delta tiller	環把套管+披覆加工-左/888A#1
8-3	101907-80501	1	Loop handle bar	Right,888A#1,Delta tiller	環把套管+披覆加工-右/888A#1
8-4	5004-08008-440	4	Set Screw	M8*1.25*8L, SUS	無頭螺栓 (耐落)/M8*1.25*8L (SUS)
			-----*-----		
8-5	5001-06035-720	4	Hex head screw	M6*P1.0*35L,Black Galvanized	外六角鋼栓(12.9級)/M6*P1.0*35L (鍍黑鋅) 半牙
8-6	5012-06010-702	4	Nylon lock nut	M6*P1.0*10*8t,Black Galvanized	尼龍螺帽/M6*P1.0*10邊*8t (鍍黑鋅)
8-7	4001308-01-A2C	1	Bracket for Head Light	A308-1, A2	前燈固定座\A308-1-A2
8-8	108101-88104	1	Head light assembly	E-1	眼鏡燈總成/E-1(FA-51)(含螺帽)
			-----*-----		
8-9	800907-881-13	1	Handle Bar Joint Base Assembly	w/ Gas Lift	把手連接座含氣壓棒模組/出配件用
-----	101201-89100-A2	1	Handle Bar Joint Base Kit	#6, for Gas Lift, A2	把手連接座/#6(配合氣壓棒)(A2)
-----	202951-89100-A2	1	Gas Lift Kit	Stroke 140, A2	氣壓棒/行程140(A2)
			-----*-----		
8-10	4003264-61	1	Spacer	C264-61	間隔圈/C264-61
8-11	5001-08040-710	1	Hex Head Screw	M8*P1.25*40L, Black Galvanized	外六角螺栓/M8*P1.25*40L (鍍黑鋅)
8-12	5021-08016-710	1	Flat Washer	8*16*1.5T,Black Galvanized	平墊圈/φ8*φ16*1.5t(鍍黑鋅)
8-13	5012-08013-700	1	Nyloc Nut	M8*P1.25*13,7.8t,Black Galvanized	尼龍螺帽/M8*P1.25*13邊7.8t(鍍黑鋅)
8-14	202912-89100	1	Depressing Lever	100L, for P Bar	壓桿/100L(P把用)
8-15	209929-88801	1	Rubber Cap for Lever of Sliding Seat	ID φ 8.1*50.7L, Black	膠套(滑道拉柄用)/ID φ 8.1*50.7L(黑色)
			-----*-----		
8-16	202908-88102-A3	2	Back Mirror for T/P/C-bar	KS-800810, A3	照後鏡(含白鐵束套)/ KS-800810(T、P、C把用)(A3)

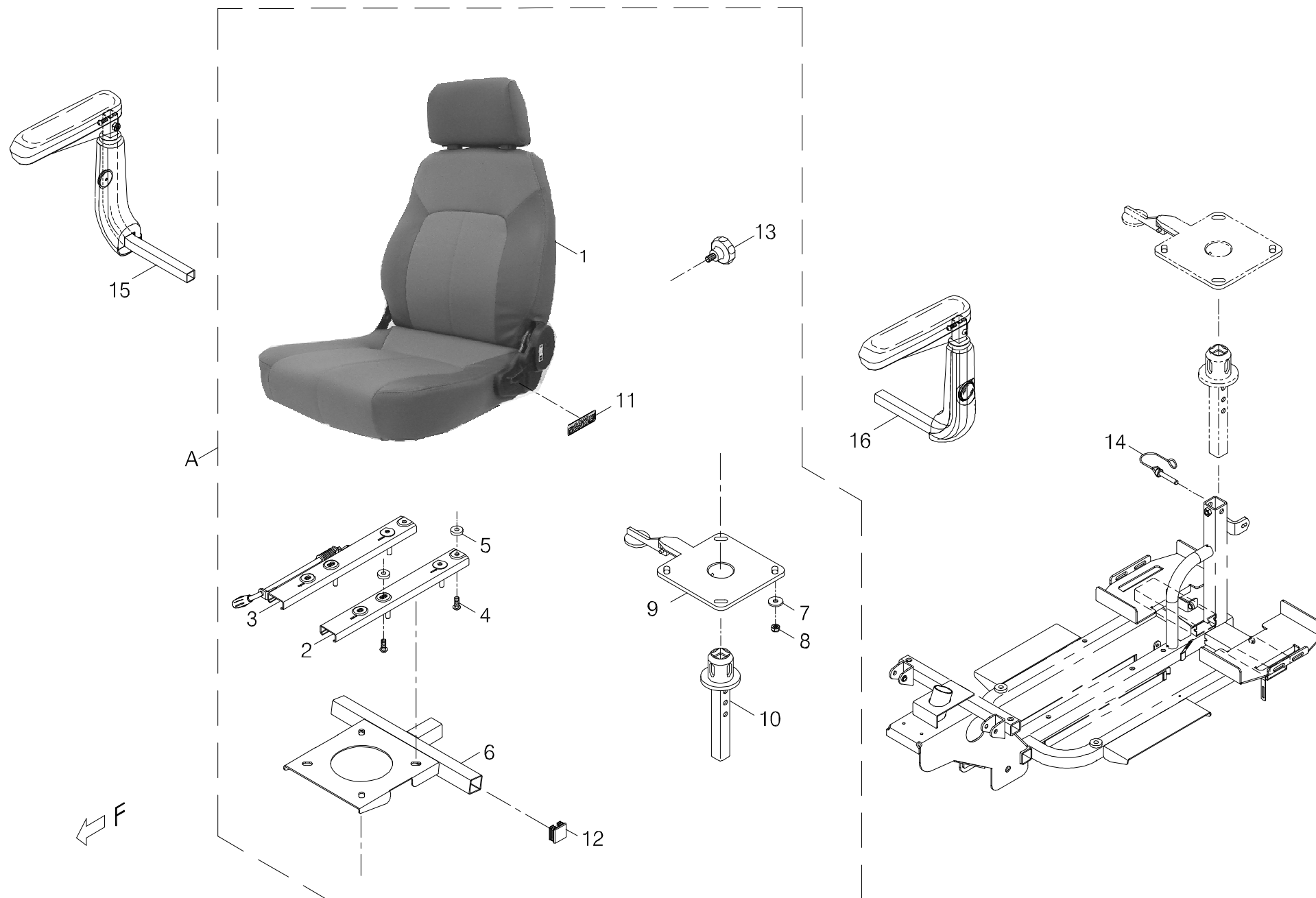


[back to contents](#)

Index NO.	Part NO.	Qty.	Description	Specification	Remark
9	-----		Swivel seat assembly		旋轉-座台總成
9-A	800317-703-01	1	Deluxe Seat Assembly w/ Sliding	Black Vinyl, Reclining, TE-9	大折疊椅模組+滑道/中背黑皮+後傾器(全包覆式)TE-9
9-1	104015-67201	1	Captain Seat Assembly	Black Vinyl, w/ Reclining	皮面中背椅+後傾DL/黑孔中黑邊(全包覆)
9-2	204901-88803-A2	1	Sliding Rail	Left, #5, A2	滑道(左)/#5(A2)
9-3	204902-88803-A2	1	Sliding Rail	Right, #5, A2	滑道(右)/#5(A2)
9-4	5003-08020-221	4	Allen Button Head Screw	M8*P1.25*20L,Multicolored Galvanized	半圓頭內六角鋼栓10.9級/M8*P1.25*20L(防鬆全支)五彩
9-5	5021-08022-211	4	Flat Washer	Dia 8*22*5t,Multicolored Galvanized	平墊圈/ \$ 8*22*5t (五彩)
9-6	204506-66101	1	Seat Adapter	#2	座台/#2
9-7	4001428-00	4	Flat Washer	A428, \$ 8.4*26*3t	平墊圈/A428 (\$ 8.4*26*3t)
9-8	5012-08013-200	4	Nyloc Nut	M8*P1.25*13 7.8t,Multicolored Galvanized	尼龍螺帽/M8*P1.25*13邊7.8t(五彩)
9-9	204507-66103	1	Swivel Seat Stand	#4	旋轉台(鋁)#4
9-10	204508-88805-A2C	1	Seat Post Adaptor	165L, A2, w/ Hole	旋轉台基座#2/165L(4孔)A2
9-11	301174-31	1	Caution Sticker	N74-31	警告貼紙/N74-31
9-12	209931-00105	2	Square Plug	30MM*30MM,2T	方形管塞/30MM*30MM 2T管用
9-13	5008-10020-212-A2	2	Tighten Knob	M10*17L,Multicolored Galvanized,A2	星型螺栓/M10*17L全牙,五彩 (A2)
			-----*		
9-14	202922-88205	1	Connecting pin	787N	附鎖安全銷/787N
			-----*		
9-15	104400-88800	1	Armrest Assembly	A1,Right	扶手總成(右)\A1
9-16	104300-88800	1	Armrest Assembly	A1,Left	扶手總成(左)\A1

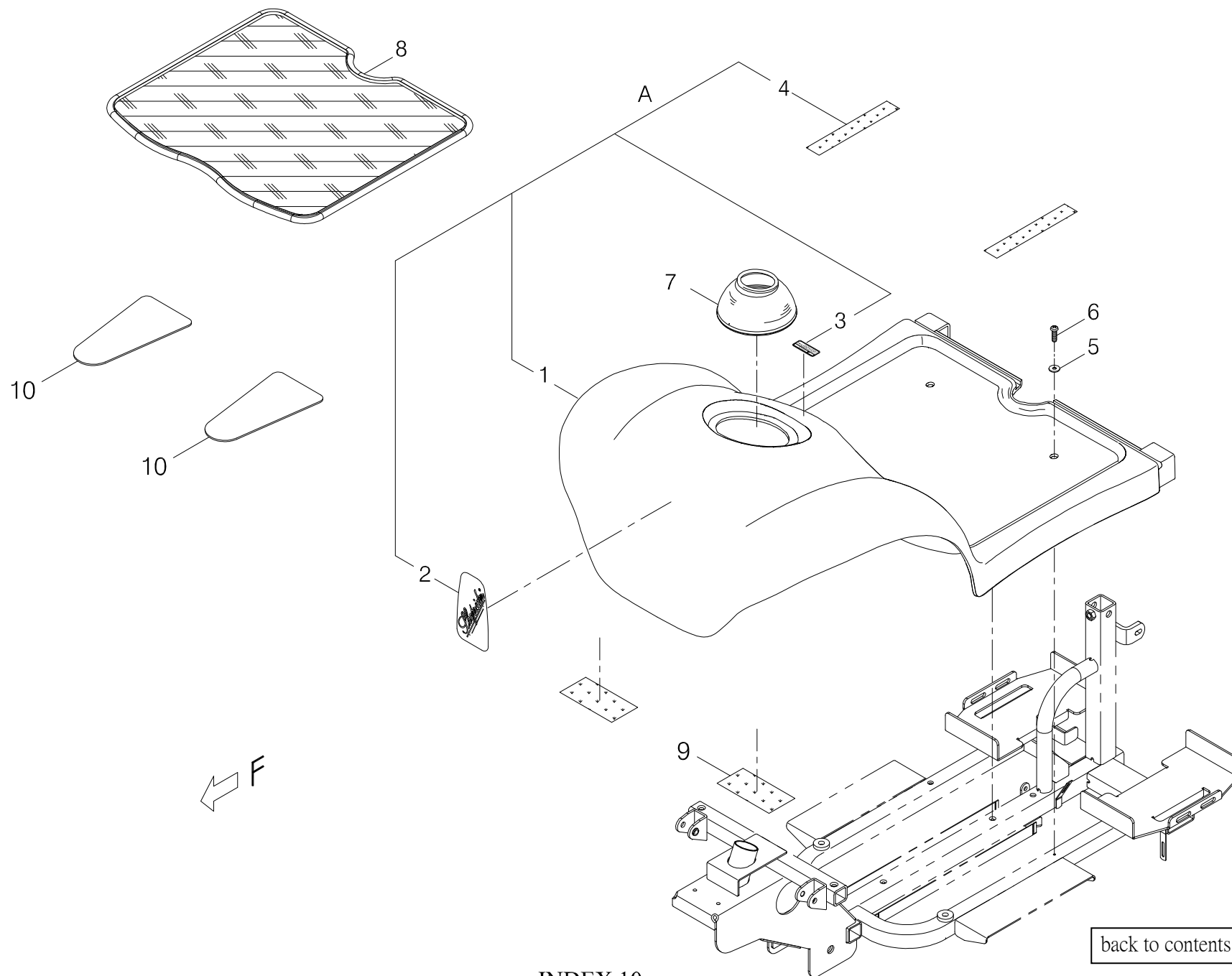

[back to contents](#)

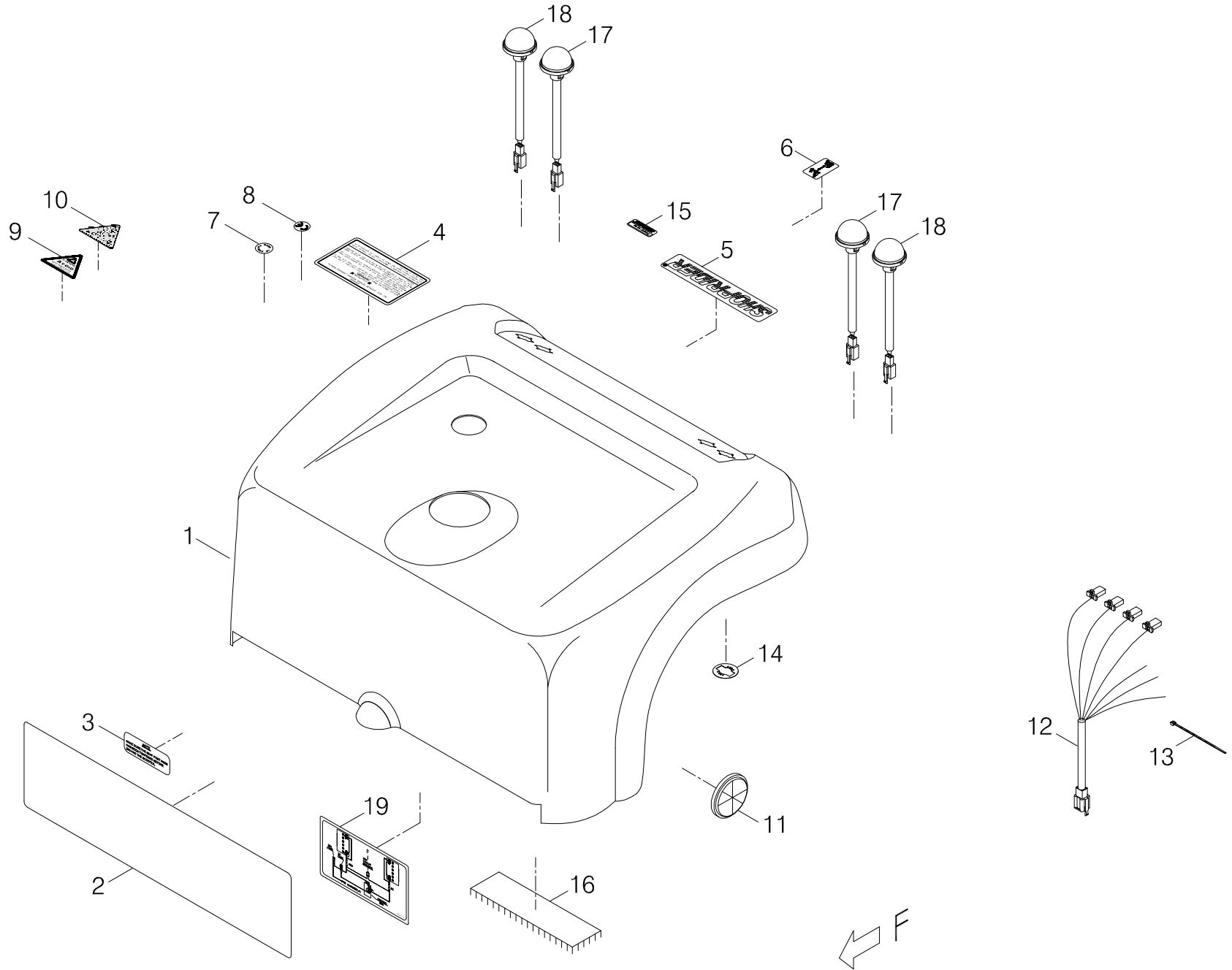


Index NO.	Part NO.	Qty.	Description	Specification	Remark
11	103202-70302-22	REF(1)	Rear Chassis Cover Assembly	w/ Decal, TE-9, Burgundy Red	後蓋板總成(含貼紙)/TE-9 棗紅
11-1	103201-70301-22	REF(1)	Rear Chassis Cover Assembly	TE-9, Burgundy Red	後蓋板總成/TE-9 棗紅(真空)
11-2	203203-89101	1	Emblem	Rear,A02,436*120mm	格紋貼紙(後)/A02(436*120)
11-3	301106-00	1	Instruction sticker	N6	指示貼紙/N6(透白)
11-4	301162-01	1	Warning sticker	N62-1	警告貼紙/N62-1(透白)
11-5	301203-00	1	Decal	T3-11,EME,175*35	商標貼紙(長型)/T3 (SHOPRIDER)175*35
11-6	301187-13	1	OFF-BRAKE-ON Label	N87-13	OFF-BRAKE-ON圖示貼紙/N87-13(透白)(代步車用)
11-7	301116-01	1	MIT Sticker	N16-1	MIT貼紙/N16-1
11-8	301166-02	1	CE Mark Label	N66-2, White Sign	CE貼紙/N66-2 透白
11-9	301123-07	1	Warning sticker	N23-7, Large	三角型警告貼紙/N23-7(大)
11-10	301123-23	1	Warning sticker	N23-23	三角型警告貼紙/N23-23
11-11	203205-88100-A2	2	Reflector	TF-301Y, Yellow(E1)	反光片(黃)/TF-301Y(E1)
11-12	207018-67200	1	Winker Cable	RA-889-N	方向燈線/RA-889-N
11-13	505201-00940-50	2	cable Tie	CV-094B	束帶/CV-094B
11-14	4001453-00	4	Snap Fastener for Lights	A453	尾燈固定片/A453
11-15	301179-04	1	Trademark Sticker	N79-4	指示貼紙(40*15.5)/N79-4(Powered by PIHSIANG)
11-16	203103-66400	REF(2)	Velcro	50W*25M, Loop	自黏扣帶(毛面)/50W*25M
11-17	108102-67101C	2	Taillight Set	Round/Plug, Red	尾燈總成/圓型/插頭(紅色)(HDY001)
11-18	108103-67101-A3C	2	Winker/Indicator	Round/Plug, Amber, FUAN 0054, A3	方向燈/圓型/插頭(琥珀色)(HDY001)(A3)
11-19	301153-06-A2	1	Battery Wiring Diagram Sticker, A2	N53-6-A2, AC2-150, 50A Circuit Breaker	指示貼紙(線路圖)/N53-6-A2 (50A)

Index NO.	Part NO.	Qty.	Description	Specification	Remark
12	-----		Handle bar cover assembly		把手蓋總成
12-1	-----	REF(1)	Front tiller cover assembly		把手上蓋總成(大)
			-----*-----		
12-2	207006-88109	1	Charging cable	KA-888-H11	充電線(短)/KA-888-H11
12-3	5009-03010-440	2	Taper Head Philip Screw	M3*P0.5*10L,SUS	埋頭螺絲/M3*P0.5*10L SUS
12-4	5012-03006-200	2	Nyloc Nut	M3*P0.5*6,Multicolored Galvanized	尼龍螺帽/M3*P0.5*6邊(五彩)
			-----*-----		
12-5	202921-88100	2	U type plate nut	Close	螺絲固定片/(閉口)
12-6	5021-06018-440	2	Flat Washer	φ 6*18*1.6t,SUS	平墊圈/φ 6*18*1.6t SUS
12-7	5005-06016-440	2	Philip Cap Head Bolt	M6*P1.0*16L,SUS	十字圓頭螺絲/M6*P1.0*16L SUS
			-----*-----		
12-8	4001151-02	1	Basket Holder	A151-2	籃子掛勾/A151-2(電著)
12-9	5005-06016-440	2	Philip Cap Head Bolt	M6*P1.0*16L,SUS	十字圓頭螺絲/M6*P1.0*16L SUS
12-10	301307-00	2	Sponge Sticker	75*30*6t	海綿貼紙/75*30*6t
12-11	108103-88104-A2	2	Winker/Indicator	Round/, Amber ,#2 FUAN 0054, A2	方向燈/圓型(琥珀色)#2 FUAN 0054(A2)
			-----*-----		
12-12	-----	REF(1)	Tiller Rear Cover	Black, w/ Hole, A2	把手下蓋/黑(加孔)(真空)(A2)
			-----*-----		
12-13	209902-77100	1	Rubber cover		把手蓋防塵套
12-14	301111-02	1	Instruction sticker	N11-2,L,SPEED,H	指示貼紙/N11-2 (L SPEED H)
12-15	209912-78500-A3	8	Plastic Stud	G3020060	腳釘/G3020060
12-16	209902-89100	1	Rubber Cover	for Gas Lift on P-Bar	把手蓋防塵套/氣壓棒+P把用
12-17	209931-89100-A2	1	Rubber Plug	for Gas Lift Kit, A2	孔塞/配合氣壓棒(A2)
12-18	301204-00	1	Decal	T4	梯型貼紙/T4 (SHOPRIDER)

Index NO.	Part NO.	Qty.	Description	Specification	Remark
			-----*-----		
12-19	301139-02	1	Instruction sticker	N39-2	指示貼紙/N39-2 (透白)
12-20	301123-14	1	Warning sticker	N23-14	三角型警告貼紙/N23-14(大)
12-21	301123-41	1	Warning Sticker	N23-41	三角型警告貼紙/N23-41
12-22	301174-05	1	Warning sticker	N74-5	警告貼紙/N74-5
12-23	301123-08	1	Warning sticker	N23-8,Small	三角型警告貼紙/N23-8 (小)
12-24	301174-16	1	Warning sticker	N74-16	警告貼紙/N74-16
12-25	301131-01	1	Instruction sticker	N31-1	指示貼紙/N31-1 (透明白字)
12-26	301123-16	1	Warning sticker	N23-16	三角型警告貼紙/N23-16(大)
12-27	301123-20	1	Warning sticker	N23-20,Front Panel	三角型警告貼紙/N23-20(大)
12-28	301123-35	1	Warning Sticker	N23-35	三角型警告貼紙/N23-35,(配合服務記錄卡)
12-29	301174-45-A2	1	Warning Sticker	N74-45, A2	警告貼紙/N74-45(A2)
12-30	301174-25	1	Warning Sticker	N74-25	警示貼紙/N74-25
12-31	301187-16	1	Instruction sticker	N87-16	指示貼紙(Lock ↑ Detach ↓)/N87-16 (透白)方型
			-----*-----		
12-A	109901-88101	1	Wicker basket assembly		籃子總成#1/竹籐(加海綿)
12-32	209916-88101-A2	1	Wicker Basket	FB-3600, Cane-like, A2	籃子(竹籐)/FB-3600
12-33	4001357-01-A2	1	Basket Bracket, Front	A357-1-A2, w/o Mark	籃子掛勾(前)/A357-1-A2(無MARK)
12-34	4001358-00-A2	1	Mounting Hook for Basket	Rear, A358-A2	籃子掛勾(後)/A358-A2
12-35	5005-06012-440	2	Screw	M6*P1.0*12L,SUS	十字圓頭螺栓/M6*P1.0*12L SUS
12-36	301307-00	1	Sponge Sticker	75*30*6t	海綿貼紙/75*30*6t




INDEX.13

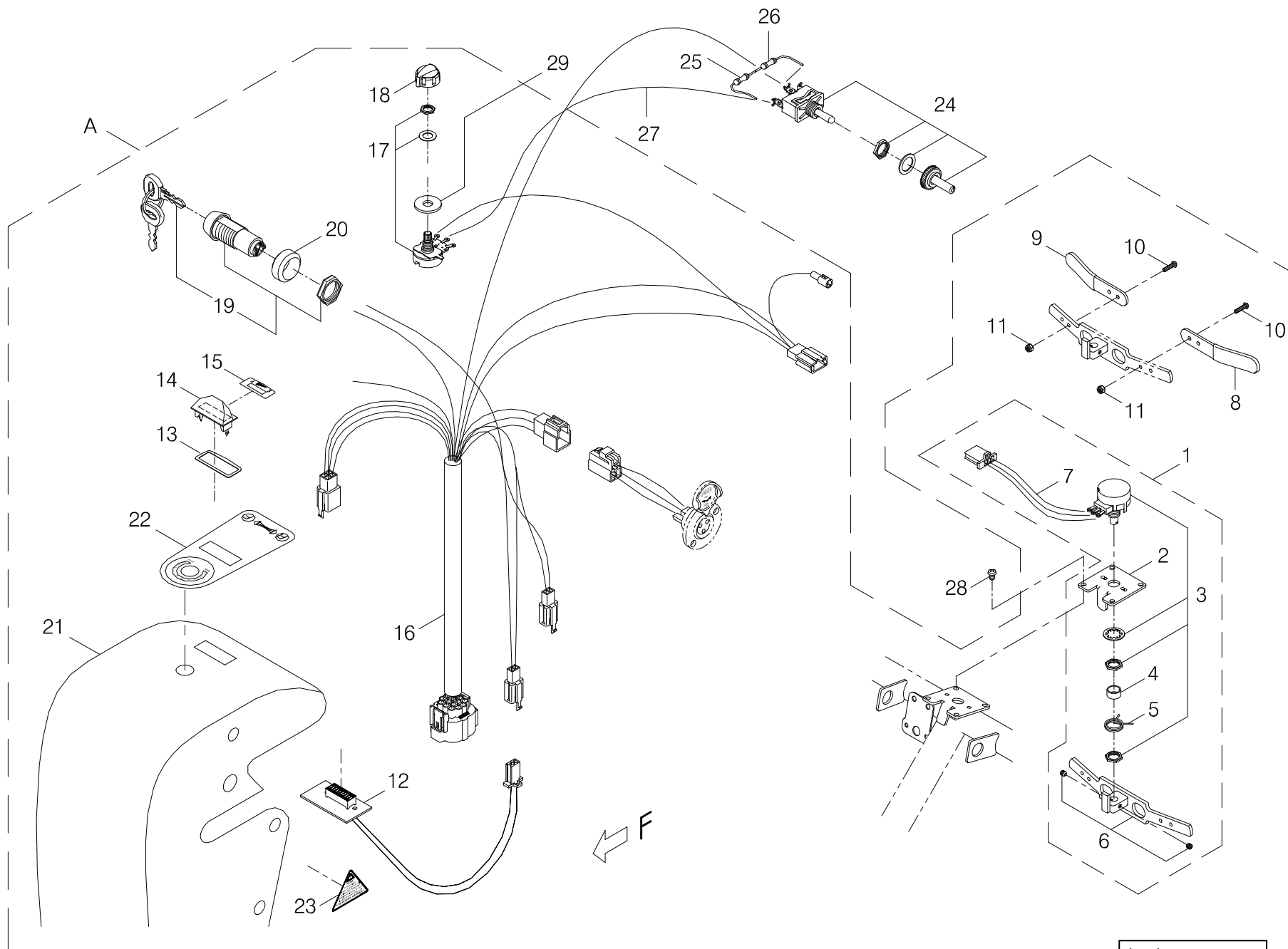


Index NO.	Part NO.	Qty.	Description	Specification	Remark
14-A	106112-88904-22	REF(1)	Upper Control Assembly, for Gas Tiller	TE-9, Burgundy Red, w/ Extension Lever	上控制組(加速桿加長片)/TE-9 棗紅 (氣壓棒用)
14-1	106108-66403-A2	1	Potentiometer	T-bar,5K#3	搖桿電阻加工/ T把(5K#3)
14-2	4001144-03	1	Potentiometer Fixing plate	A144-3, \$ 4.1	搖桿電阻固定片/A144-3 (\$ 4.1)
14-3	206103-88103-A2	1	Potentiometer	5K#3-A2	搖桿電阻/5K#3-A2(含平墊、螺帽)
14-4	4003198-00	1	Spacer	C198(\$ 11.3* \$ 9.5*5.5L)	間隔圈/C198(\$ 11.3* \$ 9.5*5.5L)
14-5	206106-88100	1	Spring		扭力彈簧
14-6	206107-88104	1	Throttle lever	#5	加速控制桿/#5(附無頭螺栓)
14-7	207010-66402	1	Control cable	KA-889-K6	上控制線/KA-889-K6
			-----*		
14-8	206122-88100	1	Extension, thottle lever	Left,,Black	加速桿加長片/左 (黑)
14-9	206123-88100	1	Extension, thottle lever	Right,,Red	加速桿加長片/右 (紅)
14-10	5005-03010-210	4	Philip Screw	M3*P0.5*10L, Multicolor Galvanized	十字圓頭螺栓/M3*P0.5*10L (五彩)
14-11	5012-03006-200	4	Nyloc Nut	M3*P0.5*6,Multicolored Galvanized	尼龍螺帽/M3*P0.5*6邊 (五彩)
			-----*		
14-12	206114-66103-A4	1	PCB for Battery Gauge	AC LED-B1-J, A4	電量指示器PC板/AC LED-B1-J(有端子)(A4)
14-13	206128-80500	1	Gasket	□34.7*12.1*0.3T	指示燈座墊片/□34.7*12.1*0.3T
14-14	206126-88100	1	Indicator lamp	Black	指示燈座/黑色
14-15	301192-00	1	Decal	N92,E,-,F	電量指示貼紙/N92 (E - F)
			-----*		
14-16	207010-80500	1	Upper Control Cable	NA-888-K1	上控制線/NA-888-K1
14-17	206109-88100-A2	1	Variable resistor	B25K(A2)	可變電阻/B25K(含平墊、螺帽)A2
14-18	206113-88101-A2	1	Speed Dial Knob	CF-X4-T	速率調整鈕/CF-X4-T(\$ 19.8)(A2)
			-----*		
14-19	106106-88103-A3-1	1	Ignition Key Assembly, Yellow	2-Way Key, 4A 125VAC 2Amp 8212, A3-1	鑰匙組總成(黃色)(含螺帽)/雙向,4A 125VAC 2Amp 8212(A3-1)

Index NO.	Part NO.	Qty.	Description	Specification	Remark
14-20	4003260-00	1	Spacer	C260(\$ 26* \$ 20*12L)	鑰匙間隔圈/C260(\$ 26* \$ 20*12L)
			-----*		
14-21	103301-66405-22-A2	REF(1)	Front Tiller Cover	X-3-XL, Burgundy Red, #1-1, A2	把手上蓋(大)/X-3-XL 棗紅-長方口#1-1(加孔)A2V
			-----*		
14-22	301220-02	1	Facia Decal	T20-2(Wood-Grainlike Texture)	上控制貼紙/T20-2(核桃木紋)
14-23	301123-30-A2	1	Note Sticker	N23-30-A2, for Sleeping Mode	三角型注意貼紙/N23-30-A2
			-----*		
14-24	208203-88101	1	Toggle switch	3P,ON-ON	搖頭開關/3P ON-ON(含防水套、墊片)
14-25	206120-89508	1	Precision Resistor	18K 1/4W	電阻(精密)/18K 1/4W
14-26	206125-66200	1	Resistor ,strict,	1K 1/4W	電阻(精密)/1K 1/4W
14-27	207012-89500	1	Jump Wire	KA-888-G4	跳線/KA-888-G4
			-----*		
14-28	5005-04006-210	2	Philip Screw	M4*P0.7*6L, Multicolor Galvanized	十字圓頭螺栓/M4*P0.7*6L (五彩)
-----	505201-02000-50	1	Cable tie	CV-200B	束帶/CV-200B
-----	9004-001	REF(1)	Heat shrink sleeve	F32,3*0.25*200M	熱縮套管(不含膠)/F32-3(3*0.25*200M)
			-----*		
14-29	5021-08022-210	1	Flat Washer	\$ 8*22*2t,Multicolored Galvanized	平墊圈/ \$ 8*22*2t (五彩)

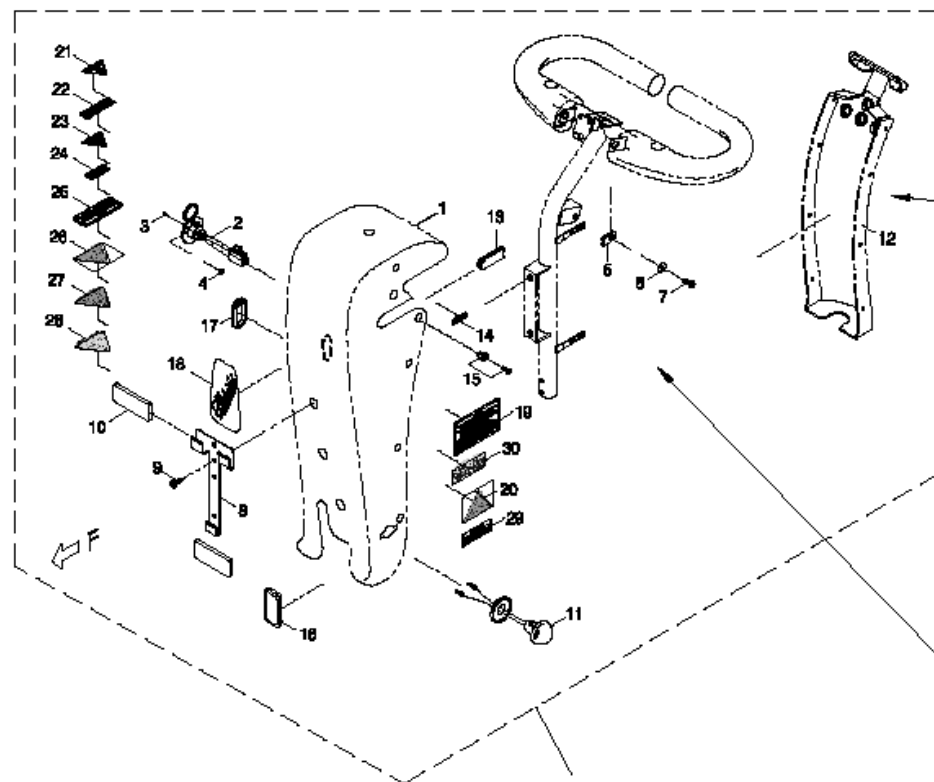


INDEX.12

INDEX.8

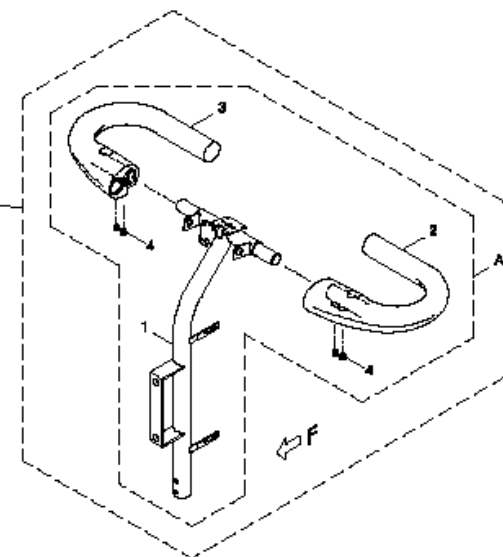


INDEX.14

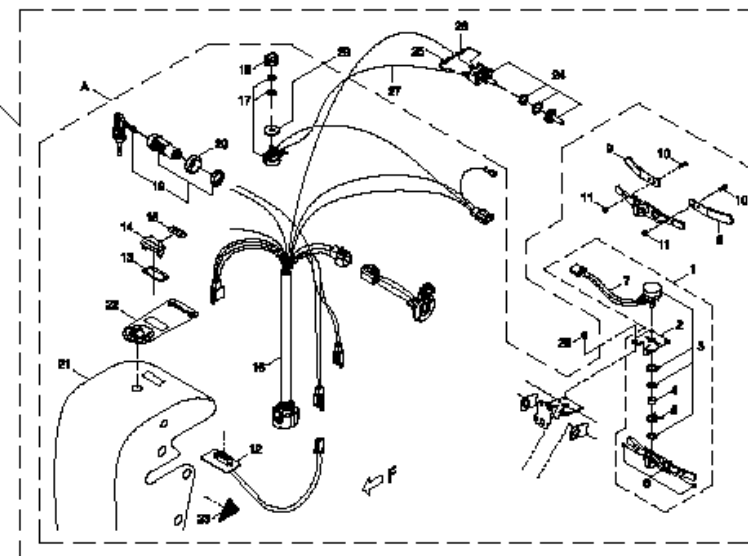


[back to contents](#)



INDEX.15

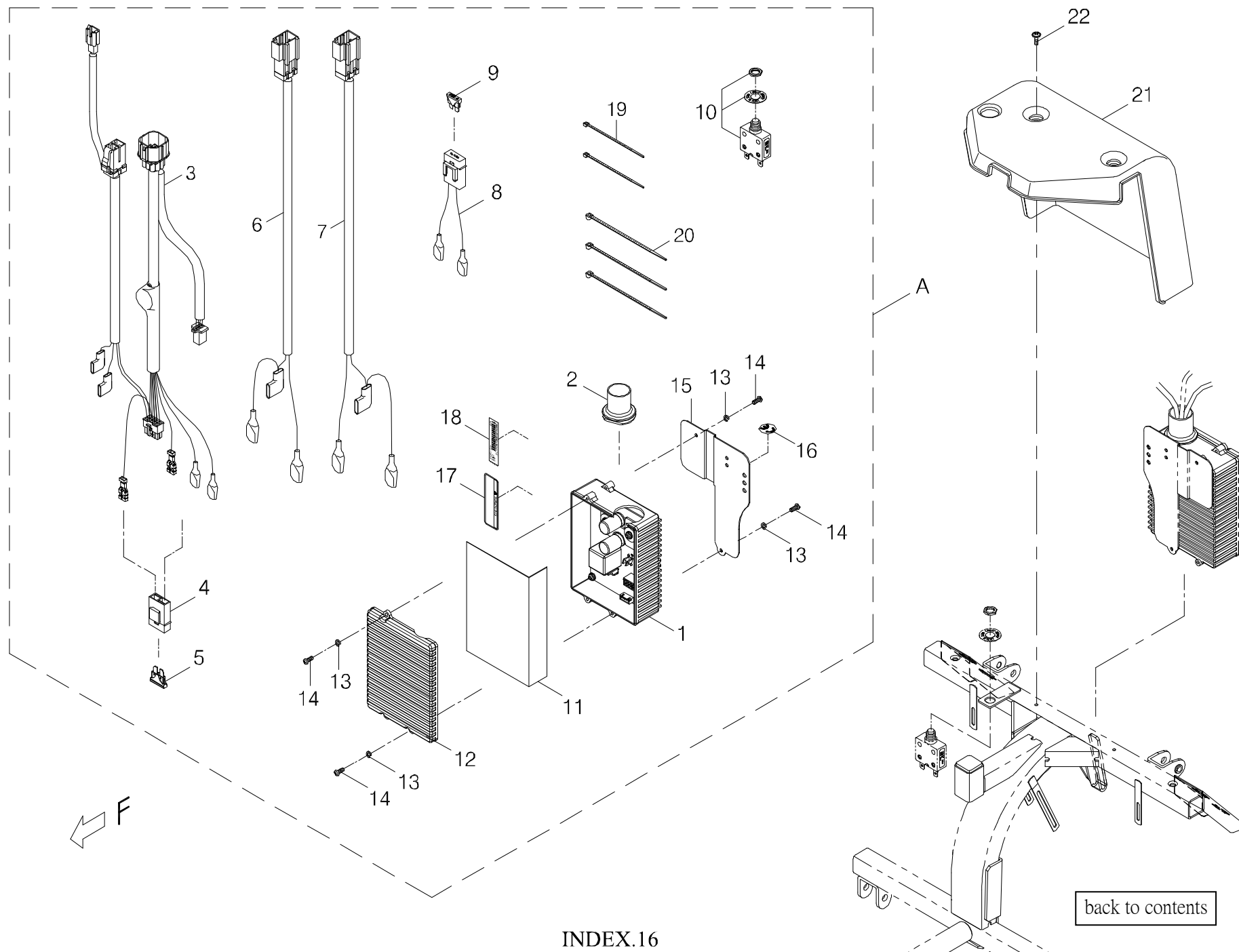

Index NO.	Part NO.	Qty.	Description	Specification	Remark
16-A	106202-88451-1	1	Main Control Box Ass'y(w/ Bracket etc.)	AC2-110, for 889NRSF w/ P-Bar	下控制總成(P把)含固定片 配件/(AC2-110)889NRSF
16-1	206221-88900-A3	1	Controller Assembly	AC2-110, w/ Case, A3	控制器/AC2-110(含盒)(A3)
16-2	206210-88103	1	Protection	AC2-110/150	護套(O)型環/AC2-110/150
16-3	207005-89824	1	Main Control Cable	NA-889-J29, w/ Horn, for P-Tiller	主控制線/NA-889-J29(含喇叭)(P把)
16-4	206206-88101	1	Fuse housing		保險絲座/盒式
16-5	206901-88100	1	Fuse	2A	保險絲 (盒式)/2A
16-6	207003-67204	1	Power Cable	+,RA-889-D4,GR889/TE-9	電源線(正)/RA-889-D4(GR889/TE-9)
16-7	207004-67204	1	Power Cable	-,RA-889-E4,GR889/TE-9	電源線(負)/RA-889-E4(GR889/TE-9)
16-8	207014-88105	1	Connection Cable	KA-888-L8	連接線/ KA-888-L8
16-9	206901-88113	1	Blade Fuse	UL 15A	保險絲 (盒式)/UL 15A
16-10	206207-88102	1	Circuit Breaker	50A	安全開關/50A(含平墊、螺帽)
16-11	302002-00	1	Insulator	152*129m/m*0.25t	絕緣膠片/152*129m/m*0.25t
16-12	206209-68500	1	Control box cover	AC2-110/150	下控制盒蓋/AC2-110/150
16-13	5022-04007-210	4	Spring Washer	§ 4*7*1.25t,Multicolored Galvanized	彈簧墊圈/ § 4*7*1.25t (五彩)
16-14	5005-04010-210	4	Philip Screw	M4*P0.7*10L,Multicolored Galvanized	十字圓頭螺栓/M4*P0.7*10L (五彩)
16-15	4001383-08-A2	1	Adaptor Bracket	A383-8, A2	控制固定片/A383-8(A2)
16-16	301166-02	1	CE Mark Label	N66-2, White Sign	CE貼紙/N66-2 透白
16-17	301174-17	1	Warning sticker	N74-17	警告貼紙/N74-17
16-18	-----	REF(1)	Decal		條碼貼紙
16-19	505201-00940-50	3	cable Tie	CV-094B	束帶/CV-094B
16-20	505201-01500-50	2	Cable Tie	CV-150B	束帶/CV-150B
			-----*-----		
16-21	203510-88801-A2C	1	Rear Decoration Cover	for 888SEL/889SEL w/ AC2, A2	後飾蓋(射出)/ 888SEL/889SEL(A2)(AC控制)
16-22	5006-04012-713	2	Self Tapping Screw w/ Washer	M4*12L, Black Galvanized	自攻螺栓 (鑽尾)帶墊/M4*12L(鍍黑鋅)

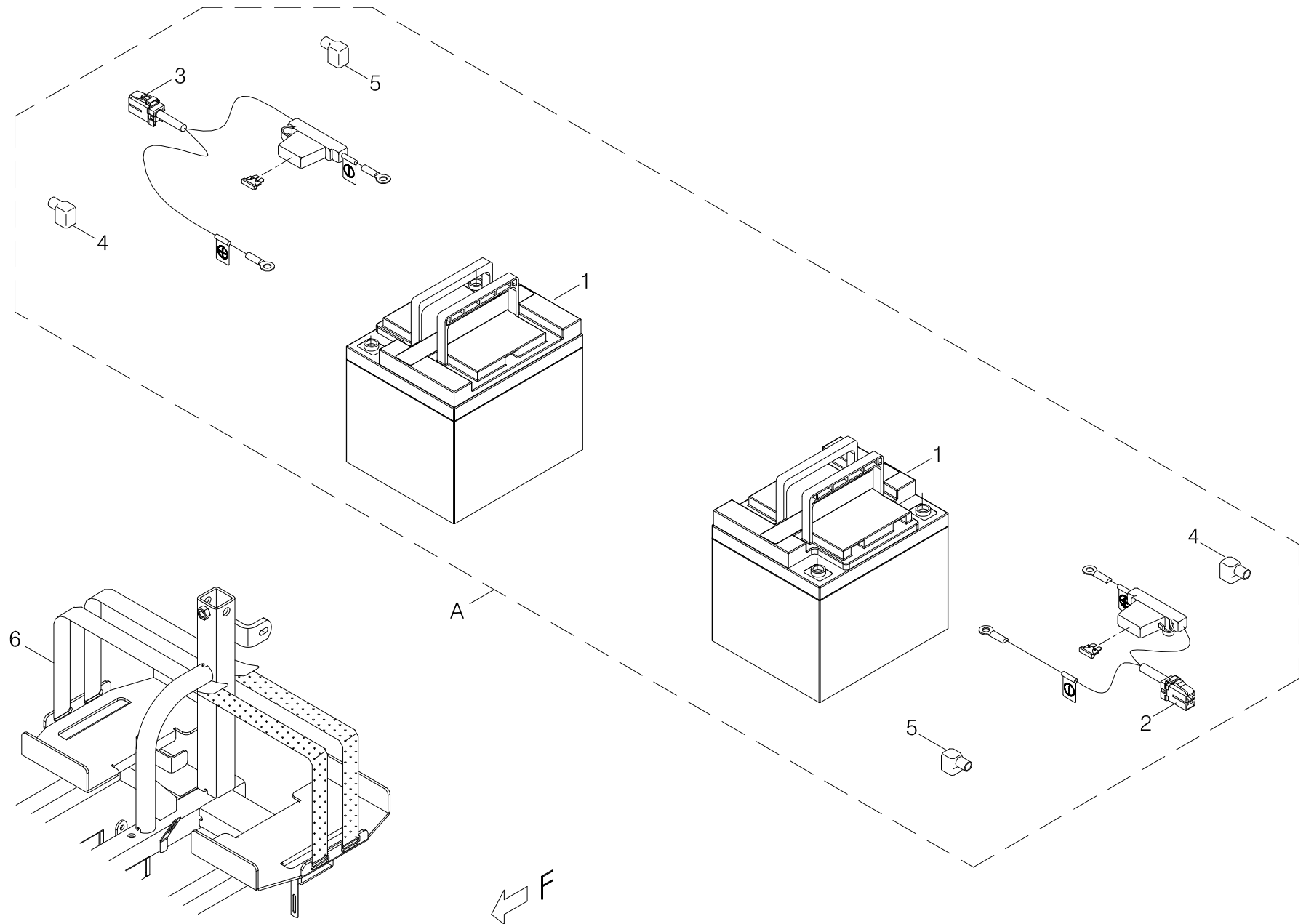

[back to contents](#)

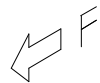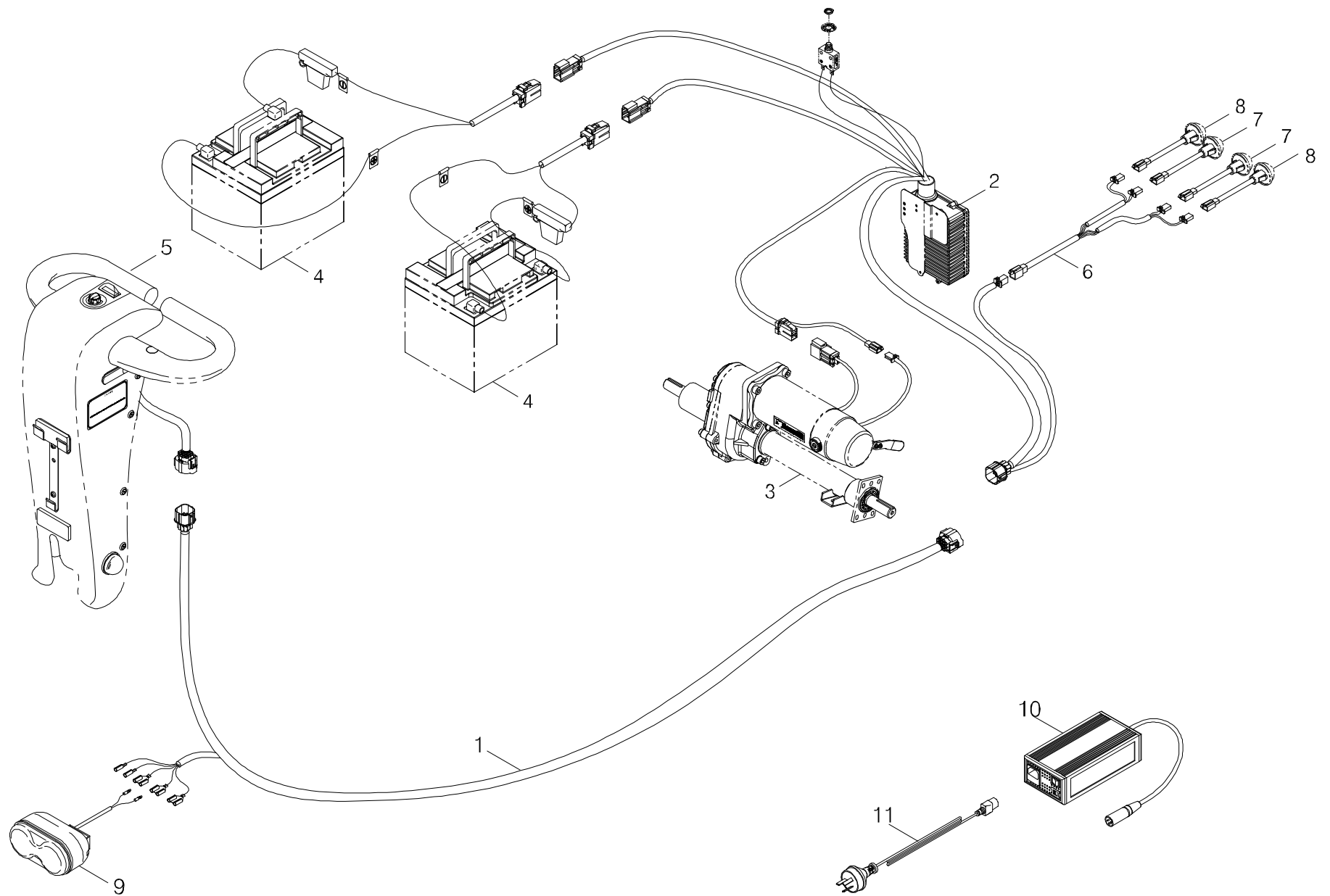


Index NO.	Part NO.	Qty.	Description	Specification	Remark
18	-----		Control and battery wiring illustration		控制及電瓶導線總成
18-1	207011-80503	1	Control connection cable	NA-888-K5	控制連接線/NA-888-K5
			-----*-----		
18-2	-----	REF(1)	Main Control Box Ass'y(w/ Bracket etc.)		下控制總成(P把)含固定片配件
			-----*-----		
18-3	-----	REF(1)	Drive-Train Assembly		馬達剎車變速箱總成
			-----*-----		
18-4	-----	REF(2)	Battery assembly w/ hand carry		電瓶總成/含手提把(L)
			-----*-----		
18-5	-----	REF(1)	P-Type Handle Bar Assembly, SHOPRIDER		P把總成(大)披覆(氣壓棒)/TE-9 棗紅(SHOPRIDER)
			-----*-----		
18-6	-----	REF(1)	Winker cable	RA-889-N	方向燈線/RA-889-N
18-7	-----	REF(2)	Tail light assembly		尾燈總成
18-8	-----	REF(2)	Winker assembly		方向燈總成
18-9	-----	REF(1)	Head light assembly		眼鏡燈總成
18-10	109201-88507-A3	1	Off-board Charger, 8A	4C24080, for AGM & GEL SLA Battery, A3	充電機(鉛酸/膠狀電瓶共用)/4C24080,8A(A3)
18-11	209201-88106	1	Power cord	Australia	充電機電源線/澳洲

Wiring Diagram for TE-9

Color Synopsis

BL:BLUE
BK:BLACK
RD:RED
YL:YELLOW
OR:ORANGE
VI:VIOLET
GR:GREEN
GY:GRAY
WH:WHITE
BR:BROWN
PK:PINK
LB:LIGHT BLUE

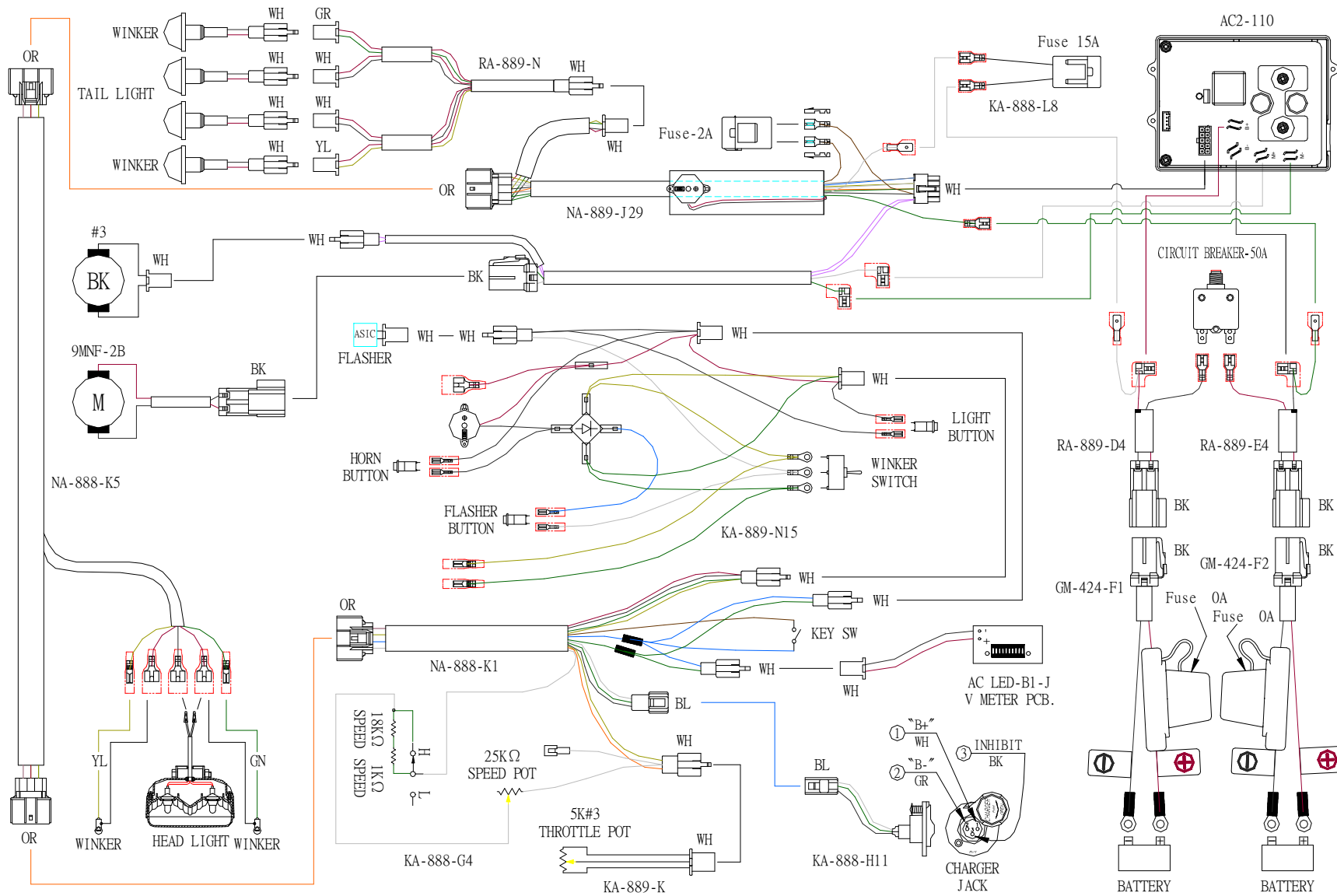

DWG.NO : AC2-110-049

EDITION : A0

[back to contents](#)